

**ACCU-CHEK® Guide Link**  
*Blood Glucose Monitoring System*

**User's Manual  
for Single Patient Use Only**

Blood Glucose Meter

**For use with compatible  
MiniMed™ 770G and  
MiniMed™ 780G systems  
with *Bluetooth®* wireless  
technology**



**Medtronic**

**Roche**



## Contents

Important Safety Information.....	2
Chapter 1: Your New Meter.....	6
Chapter 2: The Accu-Chek Softclix Lancing Device .....	11
Chapter 3: Blood Glucose Tests.....	13
Chapter 4: Control Tests.....	23
Chapter 5: Meter Settings.....	27
Chapter 6: Review Your Data.....	38
Chapter 7: Wireless Communication and Meter Pairing .....	44
Chapter 8: Meter and Lancing Device Cleaning and Disinfecting .....	48
Chapter 9: Meter Maintenance and Troubleshooting.....	52
Chapter 10: Technical Information.....	61
Chapter 11: License and Warranty .....	65
Index .....	68

## Important Safety Information

### The Accu-Chek Guide Link System

The Accu-Chek Guide Link blood glucose monitoring system is comprised of the Accu-Chek Guide Link meter and the Accu-Chek Guide test strips.\*

The Accu-Chek Guide Link blood glucose monitoring system is intended to quantitatively measure glucose in fresh capillary whole blood from the fingertip, palm, and upper arm as an aid in monitoring the effectiveness of glucose control.

The Accu-Chek Guide Link blood glucose monitoring system is intended for in vitro diagnostic single-patient use by people with diabetes.

The Accu-Chek Guide Link blood glucose monitoring system is intended to be used by a single person and should not be shared.

This system is not for use in diagnosing or screening of diabetes mellitus and not for neonatal use. Alternative site testing should be done only during steady-state times (when glucose is not changing rapidly).

The Accu-Chek Guide control solutions are for use with the Accu-Chek Guide Link blood glucose monitoring system to check that the meters and test strips are working together properly and that the test is performing correctly.

The Accu-Chek Guide Link blood glucose monitoring system is intended to be used to wirelessly transmit glucose values to the MiniMed™ 770G and MiniMed™ 780G systems with *Bluetooth*® wireless technology through the use of Bluetooth low energy communication.

The Accu-Chek Guide Link blood glucose monitoring system includes:

the Accu-Chek Guide Link meter with batteries, the Accu-Chek Guide test strips,\* the Accu-Chek Guide control solutions,\* the Accu-Chek Softclix lancing device,\* the Accu-Chek Softclix lancets.\*

\*Some items may not be included in the kit. They are a separate purchase.

### WARNING



- Choking hazard. Small parts. Keep away from children under the age of 3 years.
- Keep new and used batteries away from children. Ingestion or insertion into the body may cause chemical burns, perforation of soft tissues, and death. Severe burns may occur within 2 hours of swallowing. If you think a battery might have been swallowed or placed inside any part of the body, seek medical attention immediately.
- If the battery compartment does not close securely, stop using the product and keep it away from children. Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

### NOTE

- The term “blood glucose” is used when referring to “blood sugar.”
- Sample data screens are shown throughout the manual. Your data will differ.



# Important Safety Information

---

## Limitations

- Do not use the meter at high hematocrit levels above 65 % or low hematocrit levels below 10 %.
- Not for use in diagnosis or screening of diabetes mellitus.
- Not for neonatal use.
- Abnormally high concentrations of ascorbic acid (vitamin C) resulting in blood concentrations in excess of 5 mg/dL may cause inaccurate test results. If you are not sure please check with your doctor.
- Do not use the meter system to measure blood glucose in people who are experiencing cardiovascular collapse (severe shock) or decreased peripheral blood flow.
- Do not use this system during xylose absorption test.
- Not for use on critically ill patients, patients in shock, dehydrated patients, or hyperosmolar patients.
- This system has not been tested at altitudes higher than 10,150 feet.

## WARNING

- Do not use Alternative Site Testing to calibrate a continuous glucose monitoring system.
- Do not use Alternative Site Testing to make insulin dosing calculations.
- Alternative site testing should be done only during steady-state times (when glucose is not changing rapidly).

## Need Help?

For questions, contact the Accu-Chek Customer Care Service Center toll-free at 1-800-858-8072. Hours of operation are Monday through Friday between 8:00 am and 8:00 pm eastern standard time (EST). We offer assistance in many languages. You can also visit [accu-chek.com](http://accu-chek.com) for diabetes management tools and product demonstrations.

Please register your warranty online at [accu-chek.com/register](http://accu-chek.com/register), so you receive the best customer service possible and product update news.

# Important Safety Information

## About Testing Yourself or Others

### WARNING

- DO NOT CHANGE YOUR THERAPY BASED ON A TEST RESULT THAT DOES NOT MATCH HOW YOU FEEL OR IF YOU BELIEVE THAT YOUR TEST RESULT COULD BE INCORRECT.
- It is strongly recommended to have a back-up testing method available. Failure to test could cause a delay in therapy decisions and lead to a serious medical condition. Examples of back-up testing methods include a back-up meter and test strips. Ask your healthcare professional or pharmacist about other possible back-up methods.
- If your blood glucose result does not match how you feel and you have followed the instructions in this User's Manual, follow your healthcare professional's instructions, or contact your healthcare professional.
- During normal testing, any blood glucose meter or lancing device may come in contact with blood. All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.<sup>1,2</sup>
- The meter and lancing device should never be used by more than one person. Do not share the meter and lancing device with anyone, including family members, due to the risk of infection from bloodborne pathogens.<sup>1,2</sup> Do not use on multiple patients!

- Cleaning and disinfecting the meter and lancing device destroys most, but not necessarily all, bloodborne pathogens.<sup>3</sup>
- If the meter is being operated by a second person who is providing testing assistance to the user, the meter and lancing device should be cleaned and disinfected prior to use by the second person.
- Disinfect the meter and lancing device before allowing anyone else to handle them. Do not allow anyone else to test with the meter or lancing device.
- It is important to keep the meter and lancing device clean and disinfected. For instructions on how to clean and disinfect the meter and lancing device, see the chapter Meter and Lancing Device Cleaning and Disinfecting.
- Wash hands and dry thoroughly before and after handling the meter, lancing device, or test strips.

### NOTE

- Perform a control test when you open a new test strip box or if you think that a test result is incorrect. Performing a control test lets you know that the meter and test strips are working properly.
- Refer to the test strip and control solution package inserts for additional health-related information.
- Blood glucose and BG are interchangeable and mean the same thing.

## Important Safety Information

---

### Special Information for Caregivers

- Consult your healthcare professional to determine if it is appropriate for your child to be taught how to use the meter system or any other medical products.
- Some people with diabetes do not experience symptoms of low blood glucose (hypoglycemia). Others, such as children or people who are unconscious or have certain disabilities, may not be able to communicate their symptoms to caregivers. For these reasons, do not change any therapy without first talking to a healthcare professional.

### Before You Start Testing

#### WARNING

- Carefully read and follow the instructions in the User's Manual and package inserts for the test strips and control solutions. It is very important to follow the instructions in order to avoid an incorrect test result that leads to improper therapy.
- Inspect the test strip container before using the test strips for the first time. If you see any damage to the container cap or if anything prevents the cap from closing properly, do not use the test strips. Contact the Accu-Chek Customer Care Service Center. Damaged test strips can cause inaccurate results, which could lead to improper therapy.
- The meter, test strips, and control solution are only for use outside the body (in vitro). Do not eat the test strips. Do not swallow or inject the control solution or use the control solution for any purpose other than testing the Accu-Chek Guide Link blood glucose monitoring system.

#### NOTE

The meter may prompt you to choose a language the first time you turn it on.

# 1 Your New Meter

## The Accu-Chek Guide Link Meter



### 1. Display

Shows results, messages, and test results stored in memory.

### 2. Back Button

Returns to a previous display or field.

### 3. Up Arrow and Down Arrow Buttons

Press to move between menu options or to increase or decrease numbers.

### 4. Power/Set/OK Button

Turns meter on or off and sets options.

### 5. Test Strip Slot with Light

Insert test strip here.

### 6. Battery Door

Flip open to replace batteries.

### 7. Micro USB Port

USB functionality is not available for this meter.

### 8. Test Strip Ejector

Press to remove test strip.

# 1 Your New Meter



**9. Test Strip Container\***  
(for example)

**10. Test Strip\* – Metallic End**  
Insert this end into meter.

**11. Test Strip\* – Yellow Edge**  
Touch blood drop or control solution here.

**12. Control Solution Bottle\***

**13. Batteries**

## The Accu-Chek Softclix Lancing Device



**1. Cap with Comfort Dial Depth Selection**  
For fingertip testing.

**2. Ejector**  
Slide to release lancet.

**3. Release Button**  
Press to prick.

**4. Priming Button**  
Press to prime.

**5. Lancet**

### Note






Some items may be sold separately.

\*Some items may not be included in the kit. They are a separate purchase. Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.


# 1 Your New Meter

## Button Functions

Here are the functions of the back, arrow, and Power/Set/OK buttons on the meter. These functions are used throughout this manual. See the chapter Meter Settings for specific instructions on setting up the meter.


Button	Function
 Back Button	Return to the previous display. Return to the previous field.
  Up Arrow and Down Arrow Buttons	Navigate up and down in a menu. Increase or decrease a number.
 Power/Set/OK Button	Press briefly to turn the meter on. Press and <b>hold</b> to turn the meter off. Press to select an option. Press to move to the next field or display. Press to save an option. With the meter off, press and <b>hold</b> to check the meter display. <div data-bbox="285 940 443 1110"></div>

**NOTE**

Before you begin testing, turn the meter on by briefly pressing  so the meter can synchronize its time and date with the paired pump.

# 1 Your New Meter

## Meter Menus

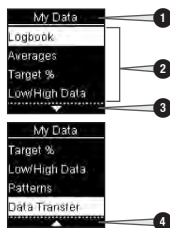

1. Title of screen or menu
2. Menu options
3. Scroll down
4. Scroll up

Display	Description
	There are more menu options listed beneath the last option. Press  on the meter to view the options.
	There are more menu options listed above the first option. Press  on the meter to view the options.
 	There are more menu options listed above and below the options. Press  or  on the meter to view the options.
	Highlighted option ( <b>Test</b> ) Press  to enter the <b>Test</b> menu.
	Highlighted field ( <b>HH=Hour</b> ) Press  or  to increase or decrease the hour. Press  to set the hour and move to the minutes field.

















## Symbols

Here is a list of the symbols on the meter display.

Symbols	Description
	Above target range
	After meal
	Bedtime

Symbols	Description
	Before meal
	Below target range
	Blood glucose test
	Checkmark / Control test OK / Selected option or setting

# 1 Your New Meter

Symbols	Description
	Control bottle
	Control test not OK
	Edit
	Error
	Fasting
	Flight mode
	Help
	Low battery
	My data
	No comment
	Other
	Overall
	Settings
	Test reminder
	Warning
	Within target range

## Set the Language


The meter may prompt you to choose a language the first time you turn it on.

1



Turn the meter on by briefly pressing **OK**. **Language** appears. Press **▼** or **▲** to highlight the language.

2

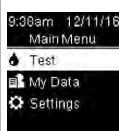

Press **OK** to set the desired language and continue.



## NOTE

If you select the wrong language and cannot correct it, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.



## 2 The Accu-Chek Softclix Lancing Device


### Using the Accu-Chek Softclix Lancing Device

	<ul style="list-style-type: none"><li>• The black cap is for fingertip testing only.</li><li>• Remove the cap by pulling it straight off. Do not twist the cap.</li></ul>
	<ul style="list-style-type: none"><li>• The penetration depth indicator on the comfort dial shows the current depth setting. The higher the number, the deeper the penetration. The best depth setting is the lowest number that lets you get enough blood for a test. Try different depth settings to find the one that is right for you.</li><li>• For soft skin, we suggest a depth setting of 2. For thick skin, try a higher depth setting.</li></ul>

### Inserting a Lancet


You must first insert a lancet into the lancing device to get it ready for use.

1

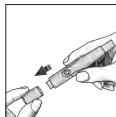


Remove the lancing device cap. Do not twist the cap.

2

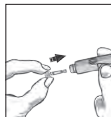


Insert a new lancet into the lancing device until it clicks.

3

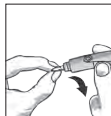

Twist off the lancet's protective cap.

4

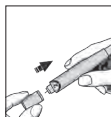

Place the cap back on the lancing device. Make sure the notch on the cap lines up with the notch on the lancing device.

## 2 The Accu-Chek Softclix Lancing Device

---

### Using the Accu-Chek Softclix Lancing Device

5



Adjust the lancet depth by turning the comfort dial.

6



Press the priming button in as far as it will go, like a pen.

The release button turns yellow when the lancing device is ready.

7

Obtain a blood drop. See the chapter Blood Glucose Tests.


#### Note

- DO NOT press the release button while pressing the priming button.
- ALWAYS use a new, sterile lancet each time you test to avoid infection.
- NEVER reuse a lancet.
- NEVER share your lancing device with anyone.

## 3 Blood Glucose Tests

### WARNING

- The meter and lancing device should never be used by more than one person. Do not share the meter and lancing device with anyone, including family members, due to the risk of infection from bloodborne pathogens.<sup>1,2</sup> Do not use on multiple patients!
- Blood glucose results can be displayed in either mg/dL or mmol/L. The back label of the meter shows the unit of measurement. If the meter shows the wrong unit, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072. The correct unit of measure in the US is mg/dL. If you do not know which unit of measurement is correct for you, contact your healthcare professional. Using the wrong unit of measurement may cause misinterpretation of your actual blood glucose level and may lead to improper therapy.

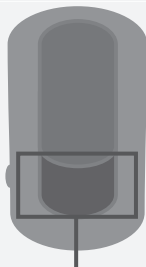

mg/dL or mmol/L is printed here

### Using the Accu-Chek Guide Link Meter System

- The Accu-Chek Guide test strips are for testing fresh capillary whole blood.
- Use only the Accu-Chek Guide test strips.
- Use the test strip immediately after removing it from the test strip container.
- Do not apply blood or control solution to the test strip before inserting it into the meter. If you applied blood or control solution before inserting the test strip into the meter, retest with a new test strip. If a result appears before applying blood or control solution, do not act on that result.
- Close the test strip container tightly immediately after removing a test strip. Moisture can damage the test strips and produce incorrect results.
- Store the unused test strips in their original container with the cap tightly closed.
- Discard the test strips if they are past the **Use By** date printed on the test strip container. If the **Use By** date is missing or cannot be read, do not use the test strips. Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.
- Refer to the test strip package insert for test strip storage and system operating conditions.
- Do not remove test strips from the test strip container and put them into another container, such as a plastic bag, pocket, purse, wallet, etc.
- Do not reuse test strips. Once control solution or blood has been applied to a test strip, discard it. If a retest is necessary, use a new test strip.
- Perform a control test every time you open a new test strip box.

## 3 Blood Glucose Tests

---

### WARNING

To prevent inaccurate results:

- If you drop the meter or drop the meter with a test strip inserted, the meter and/or test strip could be damaged. Discard the test strip and perform a control test with control solution and a new, unused test strip to ensure the meter and test strips are both working properly. Then repeat the blood glucose test with a new test strip.
- DO NOT remove test strips from the test strip container and put them into another container, such as a plastic bag, pocket, purse, wallet, etc. Storing test strips outside of the test strip container can damage the test strips and lead to inaccurate results. It is important that the test strips remain in their original container until the time of use.
- DO NOT expose test strips to heat, moisture, or humidity. Temperatures outside the required range, as well as moisture and humidity, can damage the test strips and lead to inaccurate results.
- DO NOT bend, cut, or alter the test strips.
- DO NOT get dirt, food, or other material on the test strip.
- When performing a blood glucose test, remove your finger from the test strip after the test strip is dosed and Analyzing appears on the meter.

## 3 Blood Glucose Tests

### Representation of Accuracy for Home Use

Your Accu-Chek Guide Link meter result may vary slightly from your actual blood glucose value. This is due to slight differences in technique and the natural variation in the test technology. The table and chart below show the results of a study where 120 typical users used the meter to test their blood glucose level. The system exceeds the FDA requirement of at least 95 % of results to be within  $\pm 15$  % of the reference value across the entire measuring range.

### Accu-Chek Guide System User Performance: Subject Versus Reference – Total System Capillary Accuracy

Difference range between true blood glucose level and meter result:	Meter Results Within			
	$\pm 5$ %	$\pm 10$ %	$\pm 15$ %	$\pm 20$ %
Number (and percentage) of meter results that match the true blood glucose level:	70 of 120 (58.3 %)	111 of 120 (92.5 %)	119 of 120 (99.2 %)	120 of 120 (100.0 %)
<b>Accurate Results</b> (within $\pm 15$ % of laboratory)			<b>119 of 120 results (99.2 %)</b>	
<b>More Accurate Results</b> (within $\pm 10$ % of laboratory)			<b>111 of 120 results (92.5 %)</b>	
<b>Most Accurate Results</b> (within $\pm 5$ % of laboratory)			<b>70 of 120 results (58.3 %)</b>	

### Performing a Blood Glucose Test with Blood from Your Fingertip

Refer to the Important Safety Information section at the beginning of this manual.

#### NOTE

- Before you perform your first blood glucose test, set up the meter correctly.
- You need the meter, a test strip, and a lancing device with a lancet inserted to perform a blood glucose test.
- A blood glucose test cannot be

performed while the meter is connected to a PC with a USB cable.

- There are 2 ways to start a blood glucose test.
  - Insert a test strip into the meter.
  - Turn the meter on by briefly pressing **OK**. Select **Test** > **OK**.

## 3 Blood Glucose Tests

1



Wash your hands with warm soapy water and dry thoroughly. Dirty or wet hands could affect test results.

2



Adjust the lancet depth to a comfortable penetration level.

3



Check the **Use By** date on the test strip container.

Do not use test strips past the **Use By** date.


4



Remove a test strip from the test strip container.

Close the cap tightly.

5

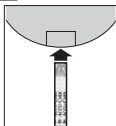

Insert the metallic end of the test strip into the meter.



The meter turns on.  
**Preparing to test** appears.

6



When **Apply drop** appears, obtain a blood drop.

7



Press the lancing device firmly against the side of your fingertip.

Press the yellow release button to prick your finger.

8



Gently squeeze your finger to assist the blood flow. This helps you get a blood drop.

## 3 Blood Glucose Tests

9




Touch the **yellow edge** of the test strip to the blood drop. Do not put blood on top of the test strip.



Remove your finger from the test strip when **Analyzing** appears.

The test result appears on the display.

**Note:** Press to automatically send result to a paired pump. If is not pressed, there will be a delay seeing the blood glucose result on the paired pump.

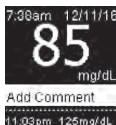

You have the option of adding a comment to the test result by pressing OR proceed to Step 12 to complete the test.


10



**Add Comment** appears. Press to highlight a comment. Press to set the comment for the test result and send the

blood glucose result to the paired pump. See the Adding Comments to Blood Glucose Results section of this chapter for details.

11

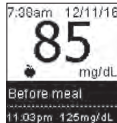

The final result appears. Press or to set the comment and return to **Main Menu**.

To change the comment, press to select the comment. Press to return to **Add Comment**.


12

Remove and discard the used test strip by pulling the test strip out of the meter or by pushing the test strip ejector on the side of the meter.



Press here to eject test strip

13

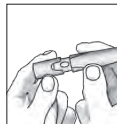


Discard the used lancet. Remove the lancing device cap.

Point the lancet away from you.

Slide out the ejector to release the lancet into a puncture-proof container. Always discard according to local regulations.

14

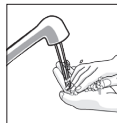

Wash hands thoroughly with soap and water.


## 3 Blood Glucose Tests

### Blood Glucose Warnings

If your blood glucose result is outside the measuring range of the meter, a warning is displayed. Press **OK** to acknowledge the LO or HI warning, OR the meter automatically moves to the LO or HI result display.



Blood glucose may be lower than the measuring range of the system. See the Unusual Blood Glucose Results section of this chapter.

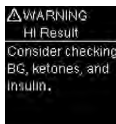

Blood glucose may be higher than the measuring range of the system. See the Unusual Blood Glucose Results section of this chapter.

### Adding Comments to Blood Glucose Results

#### NOTE

Analyzing your blood glucose results stored in the meter is an effective way for you and your healthcare professional to determine how well you are controlling your diabetes. This analysis is a valuable

tool for making improvements to your diabetes management. Use care when adding comments to blood glucose results.

#### WARNING

**CAUTION** – Incorrect comments can cause inaccurate patterns to be detected by the meter if Patterns is On.

### Overview

It is very important to have the correct time and date set in the meter. Having the correct time and date setting helps ensure accurate interpretation of information by you and your healthcare professional.

- You may add comments to blood glucose results to help you and your healthcare professional analyze patterns detected by the meter (see the Patterns section in the chapter Meter Settings for details).
- If Patterns is set to **On**, once a pattern is detected you may NOT change the comment attached to a blood glucose result (see the Patterns section in the chapter Meter Settings for details).
- Adding a comment saves the comment and the symbol with the blood glucose result.







#### NOTE

If paired with a pump, the meter time and date are automatically set to the pump's time and date during communication.




### 3 Blood Glucose Tests

Here is a list of comment symbols that can be added to a blood glucose result.

Symbol	Name	Description
	Before meal	If Patterns is <b>On</b> , select Before breakfast, lunch, dinner, or snack (see the following page for adding comments with Patterns <b>On</b> ).
	After meal	If Patterns is <b>On</b> , select After breakfast, lunch, dinner, or snack (see the following page for adding comments with Patterns <b>On</b> ).
	Fasting	Select Fasting for no caloric intake for at least 8 hours.*
	Bedtime	
	Other comment	You can use this comment to mark an event such as an AST result or exercise.
	No entry	1. You do not want to add a comment. 2. You want to remove a comment for the current blood glucose result.


\*American Diabetes Association: Standards of Medical Care in Diabetes-2016.

1




After performing a blood glucose test, the test result is displayed on the screen with **Add Comment** highlighted. Press **OK** to add a comment.

2



The **Add Comment** menu appears. Press **▼** to select the desired comment (the example here is **Before meal**). Press **OK** to select the comment.


If Patterns is On:




If **Patterns** is **On** and you select **Before meal** or **After meal**, press **▼** to select a specific meal (**Breakfast, Lunch, Dinner, or Snack**). Press **OK** to set the selected meal for the test result.

## 3 Blood Glucose Tests

3

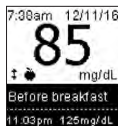

The final result appears with the comment symbol. When automatically sending test results to a paired pump, press **OK** or .

to set the comment, send the result, and return to **Main Menu**.

To change the comment, press  to select the comment.

Press **OK** to return to **Add Comment**.

### PRECAUTION

When using a test result for calibrating a continuous glucose monitoring system or to make insulin dosing calculations, confirm that the test result displayed on the pump matches the test result displayed on the meter.

## Performing a Blood Glucose Test with Blood from Your Palm or Upper Arm (Alternative Site Testing, AST)

### WARNING

- Do not use alternative site testing (AST) to calibrate a continuous glucose monitoring system.
- Do not use alternative site testing to make insulin dosing calculations.
- Alternative site testing should be done only during steady-state times (when glucose is not changing rapidly).

You have the option of obtaining a blood sample from other sites on your body besides the fingertip. Alternative sites include the palm and upper arm.

Blood obtained from the fingertip can be used at any time to test blood glucose.

If blood from an alternative site is used, there are certain times when testing is not appropriate. This is because your blood glucose level changes faster in your fingertip than in the alternative sites. These differences may cause you to misinterpret your actual blood glucose level, leading to improper therapy and potential adverse health effects.

## 3 Blood Glucose Tests

Read the next section before you try testing from alternative sites.

You may perform a palm or upper arm test	<ul style="list-style-type: none"><li>• immediately before a meal.</li><li>• while fasting.</li></ul>
You may NOT perform a palm or upper arm test	<ul style="list-style-type: none"><li>• up to 2 hours following a meal, when blood glucose values can rise quickly.</li><li>• after injecting bolus insulin, when blood glucose values can decrease rapidly.</li><li>• after exercise.</li><li>• if you are sick.</li><li>• if you think your blood glucose is low (hypoglycemia).</li><li>• if you sometimes do not notice when your blood glucose is low.</li></ul>

If you are interested in AST, talk to your healthcare professional first.

To obtain an AST cap and detailed AST instructions, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

### Unusual Blood Glucose Results

If your blood glucose result does not match how you feel, follow these steps:

1. Perform a control test. See the chapter Control Tests.
2. Repeat the blood glucose test.

If your blood glucose result still does not match how you feel, follow your healthcare professional's instructions or call your healthcare professional immediately.

#### NOTE

Always follow your healthcare professional's instructions. For example, if your healthcare professional has advised you to immediately treat a low blood glucose result (such as by eating something), then do that first.

#### WARNING

- Do not change your treatment because of one blood glucose result.
- NEVER ignore symptoms of low or high blood glucose.

## 3 Blood Glucose Tests

### Symptoms of Low or High Blood Glucose

The meter is designed to provide a numerical value for blood glucose in the range of 20–600 mg/dL.

#### WARNING

Being aware of the symptoms of low or high blood glucose can help you understand your test results and decide what to do if they seem unusual.

Low blood glucose (hypoglycemia): Symptoms of hypoglycemia may include, but are not limited to, anxiety, shakiness, sweating, headache, increased hunger, dizziness, pale skin color, sudden change in mood or irritability, fatigue, difficulty concentrating, clumsiness, palpitations, and/or confusion.

High blood glucose (hyperglycemia): Symptoms of hyperglycemia may include, but are not limited to, increased thirst, frequent urination, blurred vision, drowsiness, and/or unexplained weight loss.

If you are experiencing any of these symptoms, or other unusual symptoms, test your blood glucose from the fingertip. If your blood glucose result is displayed as LO or HI, follow your healthcare professional's instructions or contact your healthcare professional immediately. If your blood glucose result does not match how you feel, follow the steps in the Unusual Blood Glucose Results section of this chapter.

### Comparing Your Meter Result to a Laboratory Result

A common question is how the blood glucose results on the meter compare to the laboratory results. Your blood glucose can change quickly, especially after eating, taking medication, or physical activity. If you test yourself in the morning, then go to your healthcare professional's office for a blood glucose test, your test results will probably not match, even if you are fasting. This is typically not a problem with the meter, it just means that time has elapsed and your blood glucose level has changed.

Although you always apply fresh capillary whole blood to the test strip, the system has been calibrated to deliver plasma-like values for easier comparison to laboratory results.

If you want to compare your meter result to the laboratory result, **you must be fasting**. Take the meter to your healthcare professional's office and test yourself by fingerstick within 5 minutes of having blood drawn from your arm by a healthcare professional. Keep in mind that the laboratory uses different technology than the meter and that blood glucose meters for self-testing generally read somewhat lower than the laboratory result.

## 4 Control Tests

### PRECAUTION

Control results are not transmitted to the pump.

- Do not calibrate your continuous glucose monitoring device from a control result.
- Do not calculate a bolus based on a control result.

### NOTE

- Most people just test the Level 1 control. If you wish, you can also test a Level 2 control. A set of Level 1 and Level 2 control solutions is available for purchase.
- The ranges for Level 1 and Level 2 control solutions are printed on the test strip container label.
- The meter turns off 90 seconds after a successful test or 15 seconds after the test strip is removed, provided no other action is taken.

### When to Perform a Control Test

Performing a control test lets you know the meter and test strips are working properly. You should perform a control test when:

- you open a new test strip box.
- you left the test strip container open.
- you think the test strips are damaged.
- you want to check the meter and test strips.

- the test strips were stored in extreme temperatures, humidity, or both.
- you dropped the meter.
- your test result does not match how you feel.
- you want to check if you are performing the test correctly.

### About the Control Solutions

- Use only the Accu-Chek Guide control solutions.
- Close the control solution bottle tightly after use.
- Write the date you open the control solution bottle on the bottle label. The control solution must be discarded 3 months from the date the control solution bottle was opened (discard date) or on the **Use By** date on the bottle label, whichever comes first.
- Do not use control solution that is past the **Use By** or discard date.
- Refer to the control solution package insert for control solution storage conditions.
- The meter automatically recognizes the difference between the control solution and blood.
- The control results are not displayed in memory.
- The control solution can stain fabric. Remove stains by washing with soap and water.
- Control solution is available for purchase. To order the control solution, talk to your pharmacist or visit [accu-chek.com](http://accu-chek.com) to order online.

## 4 Control Tests

### Performing a Control Test

You need the meter, a test strip, and control solution Level 1 or Level 2.

1



Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.


2



Remove a test strip from the test strip container.

Close the cap tightly.

3

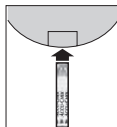

Insert the metallic end of the test strip into the meter. Place the meter on a flat surface.

Test



Preparing to test

The meter turns on. **Preparing to test** appears.

Test



Apply drop

**Apply drop** appears.

4




Select the control solution to test. You will enter the level later in the test.

5



Remove the bottle cap. Wipe the tip of the bottle with a tissue. Squeeze the bottle until a tiny drop forms at the tip.

6

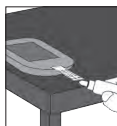

Touch the drop to the **yellow edge** of the test strip. Do not put control solution on top of the test strip.

Test


Remove finger



Analyzing

**Analyzing** appears when there is enough control solution in the test strip.

7

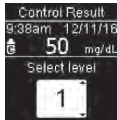

**Control Result** and the control bottle symbol appear. Press or to select the control level you tested. If you do not select a level, the control result is saved without a control level. Press .

## 4 Control Tests

8



**Within range** and ✓  
appear if the control  
result is within range.



**Out of range** and ✗  
appear if the control  
result is out of range.

9



Wipe the tip of the  
bottle with a tissue.  
Cap the bottle tightly.

Remove and discard  
the used test strip.

## 4 Control Tests

### Understanding Out-of-Range Control Results

#### WARNING

The control range applies only to a control result. It only indicates that the test strips and meter are working properly. Do not use a control result to interpret blood glucose results.

If the control result is out of range, do not use the meter until you solve the problem. Check this list to help solve the problem.

Troubleshooting Checks	Action
Were the test strips or control solutions expired?	Discard the test strips or control solution if either is past the <b>Use By</b> date. If the control solution was opened more than 3 months ago, discard it. Repeat the control test with an unexpired test strip and an unexpired control solution.
Did you wipe the tip of the control solution bottle before use?	Wipe the tip of the bottle with a tissue. Repeat the control test with a new test strip and a fresh drop of control solution.
Were the caps on the test strip container and the control solution bottle always closed tightly?	Replace the test strips or control solution if you think either was uncapped for some time. Repeat the control test.
Was the test strip used immediately after it was removed from the test strip container?	Repeat the control test with a new test strip and a fresh drop of control solution.
Were the test strips and control solutions stored in a cool, dry place?	Repeat the control test with a properly stored test strip or control solution.
Did you follow the directions?	Read the chapter Control Tests and repeat the control test.
Did you choose the correct control solution level, either 1 or 2, when you performed the control test?	If you chose the wrong control solution level, you can still compare the control result to the range printed on the test strip container.
Are you still unsure of the problem?	Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.



## 5 Meter Settings

### Overview

You can adjust the following settings in the meter for your personal preferences. Refer to the sections later in this chapter for details and how to set the options.

Setting	Options	Function
Time/Date	Time / Date	Set the time and date. <div><b>NOTE</b> Feature is only available if the meter is not paired with a pump. If paired with a pump, the meter time and date are automatically set to the pump's time and date during communication.</div>
Beeper	On / Off	Select On or Off.
Wireless	Select wireless communication settings. See the chapter Wireless Communication and Meter Pairing.	

## 5 Meter Settings

Setting	Options	Function
Target Ranges	Off / Single Range / 2 Ranges	<p>Select the blood glucose target range appropriate for you.</p> <div> <p><b>NOTE</b></p> <p>Consult your healthcare professional for the appropriate target range for you.</p> </div> <p>Off – no target range arrow symbols appear with the blood glucose result. Patterns is <b>Off</b> (see the Patterns section of this chapter for details).</p>
	70–180 mg/dL (pre-set target range)	Single Range – blood glucose results are marked as above, within, or below range based on the single target range set in the meter.
	Before Meal Range 70–110 mg/dL (pre-set target range)  After Meal Range 70–180 mg/dL (pre-set target range)	2 Ranges – set Before Meal and After Meal ranges. Blood glucose results are marked as above, within, or below range based on the 2 target ranges (Before Meal and After Meal) set in the meter.
Patterns	On / Off	On – a pattern is detected when 2 below-target or 3 above-target results with the same comment are detected within a 7-day period.
Reminders	On / Off / Edit time	On – set up to 4 reminders per day to remind you to test.
Post Meal	On / Off / Edit time	On – reminds you to perform an after meal blood glucose test.
Last Result	On / Off	<p>Select whether the previous blood glucose result (within the past 24 hours) appears with the current blood glucose result.</p> <p>On – the previous blood glucose result appears with the current blood glucose result.</p> <p>Off – only the current blood glucose result appears.</p>
Language		Select the language for the meter.

## 5 Meter Settings

### Time/Date

#### NOTE

Feature is only available if the meter is not paired with a pump. If paired with a pump, the meter time and date are automatically set to the pump's time and date during communication.

1



Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2



**Time/Date** is highlighted. Press **OK**.

3



Press **▼** or **▲** to adjust each field. Press **OK** to set and move to the next field. Set **am** or **pm** if necessary.

Press **OK** to save and return to the previous menu.

### Beeper

The beeper prompts you:

- when a test strip is inserted.
- to apply blood or control solution to the test strip.
- when enough blood or control solution is drawn into the test strip.
- when the blood glucose or control test is complete.
- when the meter is turned on.
- when a button is pressed.
- when it is time to perform a test (if Reminders or Post Meal reminders are **On**).
- when the batteries are inserted.
- when there are no stored blood glucose results or there is an invalid record in the logbook.
- when there are no errors in the error log.
- if an error occurred (even if the beeper is off, it still beeps for an error).

1



Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.


2



Press **▼** to highlight **Beeper**. Press **OK**.

## 5 Meter Settings

3

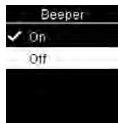

Press or to highlight **On** or **Off**. Press **OK** to move to the option.

Press **OK** to set the option and return to the previous menu.

### Target Ranges

Your healthcare professional can tell you what blood glucose range is appropriate for you. It is very important to stay within your target range.

Target Ranges can be set from a lower limit of 60–100 mg/dL to an upper limit of 101–300 mg/dL.

Options	Function
Off	Arrow symbols for above, within, or below target ranges do not appear with the blood glucose results.
Single Range	Set lower limit and upper limit for the Target Range.  You will be prompted to turn on Patterns if desired (see the Patterns section of this chapter for details).

2 Ranges

Set lower limits and upper limits for Before Meal and After Meal target ranges.

You must mark your blood glucose results with a comment for the meter to detect above, within, or below Before Meal or After Meal test results (see the Adding Comments to Blood Glucose Results section in the chapter Blood Glucose Tests for details).

You will be prompted to turn on Patterns if desired (see the Patterns section of this chapter for details).

When Target Ranges is on, the following symbols appear with blood glucose results.

Symbol	Meaning
	The blood glucose result is below the target range.
	The blood glucose result is within the target range.
	The blood glucose result is above the target range.

### WARNING

This function is no substitute for hypoglycemia training by your healthcare professional.

## 5 Meter Settings

1

9:00am 12/11/16  
Main Menu  
Test  
My Data  
Settings

Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2

Settings  
Time/Date  
Beeper  
Wireless  
More Options

Press **▼** to highlight **More Options**. Press **OK**.

3

More Options  
Target Ranges  
Patterns  
Reminders  
Post Meal

**Target Ranges** is highlighted. Press **OK**.

4

Target Ranges  
✓ Off  
Single Range  
2 Ranges  
Help

Press **▼** or **▲** to highlight **Off**, **Single Range**, or **2 Ranges**. Press **OK** to select an option (the example here is **2 Ranges**).

5

Target Ranges  
Before Meal  
70 - 130  
After Meal  
70 - 180  
mg/dL

Press **▼** or **▲** to adjust the lower limit of the **Before Meal** target range. Press **OK** to set and move to the next field.

Continue to set the upper limit for the **Before Meal** target range and the lower and upper limits for the **After Meal** target range. Press **OK**.

6

Target Ranges  
Off  
Single Range  
✓ 2 Ranges  
Help

The **✓** appears next to the option you selected.

Press **↶** to return to the previous menu.

Target Ranges  
Would you like to turn on Patterns?  
No  
Yes  
Help

**The meter may prompt you to turn on Patterns.**

Press **▼** or **▲** to highlight **Yes** or **No**.

Press **OK** to set the option and return to the previous menu (see the Patterns section of this chapter for details).

## 5 Meter Settings

### NOTE

Analyzing your blood glucose results stored in the meter is an effective way for you and your healthcare professional to determine how well you are controlling your diabetes. This analysis is a valuable tool for making improvements to your diabetes management. Use care when adding comments to blood glucose results. Incorrect comments can cause inaccurate patterns to be detected by the meter if Patterns is **On**.

### Patterns


A Pattern is detected by the meter when 2 below-target (Low Pattern) or 3 above-target (High Pattern) test results with the same comment are detected within a 7-day period.

- The meter does NOT detect a pattern for the “other” comment added to blood glucose results.
- It is very important to have the correct time and date set in the meter. Having the correct time and date setting helps ensure accurate interpretation of information by you and your healthcare professional.
- Only blood glucose results marked with comments will be included in Patterns. If LO or HI test results are marked with comments, the test results become part of Patterns (see the chapter Blood Glucose Tests for more details).
- Target Ranges must be set in the meter to use Patterns. If Target Ranges is not set, the meter prompts you to set it.

### NOTE

If paired with a pump, the meter time and date are automatically set to the pump's time and date during communication.

1

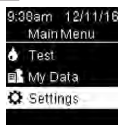

Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

2



Press **▼** to highlight **More Options**. Press **OK**.


3




Press **▼** to highlight **Patterns**. Press **OK**.

## 5 Meter Settings

4

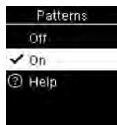


Press to highlight **On**. Press to move to the option. Press .

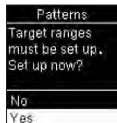

**This message appears if Target Ranges is Off:**

Press to highlight **Yes**. Press .

(To turn Patterns **Off**, select **No**. Press to return to **Patterns**.)

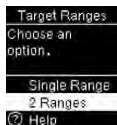


**Target Ranges** appears on the display (see the Target Ranges section of this chapter for details on setting target ranges).

### NOTE

When automatically sending blood glucose results to a paired pump, select the **View later** option to send the test result without delay or press to automatically send result to a paired pump.

If Patterns is **On** and a new pattern is detected with a blood glucose result, a message appears on the display.

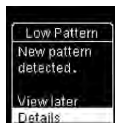

Press to select **Details** to view the blood glucose results that make up that pattern.

or




Press to highlight **View later**.

Press to return to the previous screen.

# 5 Meter Settings

## Reminders

You can set up to 4 general test Reminders per day to remind you to test. A series of beeps sound and  is displayed for Reminders set in the meter.


Reminders:

- turn off by inserting a test strip or pressing any button.
- are postponed until the next test reminder if a test was performed within 15 minutes of a test reminder.
- do not appear/beep if the meter is on at the test reminder time.
- do not appear/beep if the meter is connected and communicating to the paired pump.
- do not beep if the meter beeper is set to off.
- do not appear/beep if the batteries need to be replaced.

## Set Reminders

- Reminder times are pre-set in the meter for 8:00am, 12:00pm, 6:00pm, and 10:00pm. You may change reminder times by following the instructions below.
- If a general test Reminder is set for the same time as a Post Meal reminder, the Post Meal reminder will appear/beep instead of the general Reminder (see the Post Meal Reminders section of this chapter for details).

**2**




Press  to highlight **More Options**. Press **OK**.


**3**




Press  or  to highlight **Reminders**. Press **OK**.


**4**



Pre-set reminder times appear on the display. Press  to highlight a reminder time. Press **OK**.

**1**

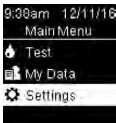


Turn the meter on by briefly pressing **OK**. From **Main Menu**, press  to highlight **Settings**. Press **OK**.




## 5 Meter Settings

5

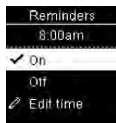

The pre-set reminder time appears. Press or to select **On** or **Off**. Press to move to the option.

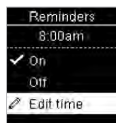


### To change the Reminder time shown:

Press to highlight **Edit time**. Press .



Press or to adjust each field. Press to move to the next field. Press to return to the previous menu.

6

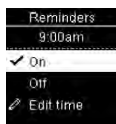

A indicates the reminder is set for the time shown and automatically set to **On**.

To turn the reminder off, press to highlight **Off**. Press .

Press to return to **Reminders**.

7



The reminder time appears.

Press to continue to set additional reminders or press to return to **More Options**.

## Post Meal Reminders

Post Meal reminders can be set to remind you to test again later when you add a Before meal comment to a blood glucose result. When a reminder occurs, a series of beeps sound and is displayed.

Post Meal reminders:


- turn off by inserting a test strip or pressing any button.
- are postponed until the next test reminder if a test was performed within 15 minutes of a test reminder.
- do not appear/beep if the meter is on at the test reminder time.
- do not appear/beep if the meter is connected and communicating to the paired pump.
- do not beep if the meter beeper is set to off.
- do not appear/beep if the batteries need to be replaced.

## Set Post Meal Reminders

- Adding a Before meal comment to a blood glucose result sets a Post Meal reminder in the meter.
- Marking blood glucose results with an After meal comment provides more information about your test results to help you and your healthcare professional in the management of your diabetes.
- Talk to your healthcare professional to determine your Post Meal test time.
- Select 1 hour, 1.5 hours, or 2 hours for Post Meal reminders to occur.

## 5 Meter Settings

**1**



Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

**2**



Press **▼** to highlight **More Options**. Press **OK**.

**3**



Press **▼** to highlight **Post Meal**. Press **OK**.

**4**




Press **▼** or **▲** to highlight **On**. Press **OK** to move **✓** to the option.

**5**



Press **▼** to highlight **Edit time**. Press **OK**.

**6**




Press **▼** or **▲** to highlight **1 hour**, **1.5 hours**, or **2 hours** after a meal for **Post Meal** reminders to occur.

Press **OK** to move **✓** to the option.

Press **OK** to set and return to the previous menu.

### Last Result

Select whether the previous blood glucose result appears with the current blood glucose result. **Test results older than 24 hours do not appear.**

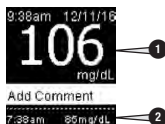

106 mg/dL


Add Comment

7:38am 85mg/dL

Off – only the current blood glucose result **1** appears.

On – the previous blood glucose result **2** appears with the current blood glucose result.

**1**



Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

## 5 Meter Settings

2




Press to highlight **More Options**. Press .

3



Press to highlight **Last Result**. Press .

4

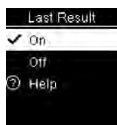


Press or to highlight **On** or **Off**. Press to move to the option.

Press to set the option and return to the previous menu.

### Language

Choose the language that appears on the meter.

1

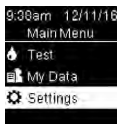

Turn the meter on by briefly pressing . From **Main Menu**, press to highlight **Settings**. Press .

2




Press to highlight **More Options**. Press .

3



Press to highlight **Language**. Press .

4

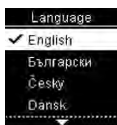

Press or to highlight the desired language. Press to move to the option.

Press to set the language and return to the previous menu.

## 6 Review Your Data

### Overview


- Blood glucose results are stored from the newest to the oldest.
- The meter automatically stores up to 720 blood glucose results in memory with the time and date of the test and any test result comments.
- Once 720 blood glucose results are in memory, adding a new blood glucose result deletes the oldest blood glucose result.
- Only test results that have been marked with a fasting, before meal, after meal, or bedtime comment are included in the average for that comment.
- All test results are included in the overall 7, 14, 30, and 90-day averages regardless of what comment is added.
- Control results are not included in the averages or blood glucose reports.

### WARNING

Do not change your therapy based on an individual test result in memory. Talk to your healthcare professional before changing therapy based on test results in memory.

### Logbook

1

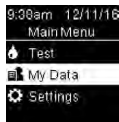

Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **My Data**. Press **OK**.

2



**Logbook** is highlighted. Press **OK**.

3



Logbook		
11/10/16		mg/dL
2:30pm	●	145
12:00pm	✱	65
8:38am	☉	106
7:38am	☾	85

Press **▼** or **▲** to scroll through **Logbook**.

The most recent test result **1**.

The 2nd most recent test result **2**.

## 6 Review Your Data

4

To view details about a test result, press or to highlight the test result. Press . Test result details shown below only appear if **Target Ranges** is on or comments were added to a test result.

### The most recent test result



### The 2nd most recent test result



## Averages

1



Turn the meter on by briefly pressing . From **Main Menu**, press to highlight **My Data**. Press .

2



Press to highlight **Averages**. Press .

3



Press to highlight a category (the example here is **Overall**). Press .

4



Press to highlight a time period (the example here is **90 days**). Press .

5



Press to return to the previous menu if you want to review a different time period OR press or to move through different averages.

## Target Percent (%)

Target Percent (%) allows you to view the percentage of your Overall, Before meal, After meal, Fasting, and Bedtime blood glucose results that are above, within, or below your target ranges.

- Target % results can be viewed for 7, 14, 30, or 90-day time periods.
- Target Ranges must be set in the meter to review Target % results.

## 6 Review Your Data

1



Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **My Data**. Press **OK**.

2



Press **▼** to highlight **Target %**. Press **OK**.

3



Press **▼** to highlight a category (the example here is **Before meal**). Press **OK** to select the option.

4



Press **▼** to highlight a time period (the example here is **90 days**). Press **OK**.

5



The **Target %** appears (for the **Before meal** example). The number of total tests included in the **Target %** appears at the bottom of the display.

Press **↶** to return to the previous menu.

### NOTE

Analyzing your blood glucose results stored in the meter is an effective way for you and your healthcare professional to determine how well you are controlling your diabetes. This analysis is a valuable tool for making improvements to your diabetes management. Use care when adding comments to blood glucose results. Incorrect comments can cause inaccurate patterns to be detected by the meter if Patterns is **On**.






### Low/High Data

Your healthcare professional can tell you what blood glucose range is appropriate for you. It is very important to stay within your target range.

- Target Ranges must be set in the meter to track Low/High Data test results (see the Target Ranges section in the chapter Meter Settings for details).
- Low BG or High BG Data includes only test results that fall above or below the target ranges set in the meter.
- Low BG or High BG Data is tracked in the meter for 30 days.


## 6 Review Your Data

You can select Low BG and High BG test results for Overall, Before meal, After meal, Fasting, or Bedtime blood glucose results.

Symbol	Name	Description
	Overall	Includes low and high blood glucose results based on Target Ranges set in the meter.
	Before meal	You may view low or high test results marked with a Before meal comment for Overall, Before breakfast, Before lunch, Before dinner, and Before snack blood glucose results.*
	After meal	You may view low or high test results marked with an After meal comment for Overall, After breakfast, After lunch, After dinner, and After snack blood glucose results.*
	Fasting	Includes low or high fasting blood glucose results marked as Fasting in comments.
	Bedtime	Includes low or high bedtime blood glucose results marked as Bedtime in comments.

\*Test results for Before and After specific meals are only available if Patterns is set to **On**.

**1**

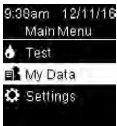


Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **My Data**. Press **OK**.

**2**



Press **▼** to highlight **Low/High Data**. Press **OK**.


**3**

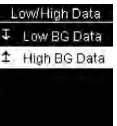

**If Target Ranges is on:**

Press **▼** to select **Low** or **High BG Data** (the example here is **High BG Data**). Press **OK**.

or



**If Target Ranges has NOT been on within the last 30 days:**

This message appears on the meter (to turn on Target Ranges see the Target Ranges section in the chapter Meter Settings for details).

## 6 Review Your Data

4



Press to highlight a category (the example here is **Before meal**). Press .



**If test results with detailed meal comments are saved in the Logbook:**

The meter may prompt you to select detailed categories to view. Press to highlight a category (the example here is before **Breakfast**). Press .

5



The selected data appears (the example here is **High BG Data**). Press to scroll through the test results.

Press to return to the previous menu.

### NOTE

Analyzing your blood glucose results stored in the meter is an effective way for you and your healthcare professional to determine how well you are controlling your diabetes. This analysis is a valuable tool for making improvements to your diabetes management. Use care when adding comments to blood glucose results. Incorrect comments can cause inaccurate patterns to be detected by the meter if Patterns is **On**.

## Patterns

- Patterns displays only active Low Patterns or High Patterns based on comments added to blood glucose results within the last 7 days.
- A Pattern is generated when 2 below-target or 3 above-target test results with the same comment are detected within a 7-day period.

Patterns may be viewed on the meter in 3 ways:




when a Patterns option is displayed at the bottom of **Main Menu**.

or



from **My Data** on **Main Menu**.

or

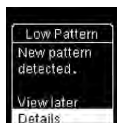

if a **New pattern detected** message appears on the display when performing a blood glucose test.




## 6 Review Your Data

Patterns detected by the meter may be displayed on the Main Menu as:

Patterns	High and low patterns have been detected	Patterns may include the following blood glucose results marked with comments:  Before breakfast, After breakfast, Before lunch, After lunch, Before dinner, After dinner, Before snack, After snack, Fasting, or Bedtime (see the Adding Comments to Blood Glucose Results section in the chapter Blood Glucose Tests).
High Pattern(s)	One or more high patterns have been detected	
Low Pattern(s)	One or more low patterns have been detected	
No Patterns	No active pattern based on test results from last 7 days	
Blank	Patterns feature is set to <b>Off</b>	


**1**



9:38am 12/11/16  
Main Menu  
Test  
My Data  
Settings  
**Patterns**

Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight the **Patterns** option at the bottom of the screen (the example here is **Patterns**). Press **OK**.


**2**

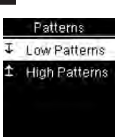

**Patterns**  
Low Patterns  
High Patterns

Press **▼** or **▲** to highlight **Low Patterns** or **High Patterns** (the example here is **Low Patterns**). Press **OK** to select the option.


**3**



Low Patterns  
Before breakfast  
Before lunch  
Before dinner  
Before snack

Press **▼** to highlight a category to review (the example here is **Before breakfast**). Press **OK**.

**4**



Low Pattern  
Before breakfast  
12/11/16 85  
7:38 am mg/dL

Press **▼** to scroll through test results. Press **↶** to return to the previous menu.

# 7 Wireless Communication and Meter Pairing

## Overview

You can wirelessly and automatically synchronize your diabetes information with the MiniMed™ 770G and MiniMed™ 780G systems with *Bluetooth®* wireless technology. The process of creating a connection between the meter and the pump is called pairing.

Setting	Options	Function
Pairing	Pair to Pump / Delete Pairing	Select whether to pair a pump or to delete the paired pump.
Flight Mode	On / Off	Select whether wireless communication is available. On – wireless communication is not available. Off – wireless communication is available.
Auto-Send	On / Off	Select whether data is automatically sent to the paired pump after each test. On – data is automatically sent to the pump. Off – data is not automatically sent to the pump.


## Pairing

The meter can automatically send blood glucose results to a MiniMed™ 770G or MiniMed™ 780G system with *Bluetooth®* wireless technology. This feature eliminates the need to manually enter your blood glucose result on the pump. Before the meter can send blood glucose results to a pump, the meter and pump must be paired. Refer to the MiniMed™ 770G or MiniMed™ 780G System User Guide for pairing instructions.

Once the meter and pump are paired, the pairing settings are stored in both devices so that you do not have to repeat the pairing. If communication between the meter and pump is stopped or interrupted for any reason, they will automatically resume communication when both devices are in communication range.

The meter can only be paired with a MiniMed™ 770G or MiniMed™ 780G system. The meter can only be paired with 1 pump at a time. It is not necessary to delete the existing pairing when pairing with a new pump. Any blood glucose results stored on the meter prior to pairing cannot be transferred from the meter to the pump.

## Flight Mode

Select whether wireless communication is available or not. When **Flight Mode** is on,  appears in the title bar and wireless communication is not available.

1


9:38am 12/11/18

Main Menu

Test


My Data

Settings

Turn the meter on by briefly pressing **OK**. From **Main Menu**, press  to highlight **Settings**. Press **OK**.


## 7 Wireless Communication and Meter Pairing

2

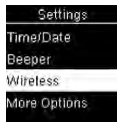

Press ▼ to highlight **Wireless**. Press **OK**.

3

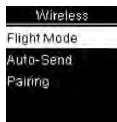


**Flight Mode** is highlighted. Press **OK**.

4



Press ▼ or ▲ to highlight **On** or **Off**. Press **OK** to move ✓ to the option.

Press **OK** to set the option and return to the previous menu.

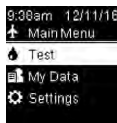

**Main Menu** with meter in Flight Mode.

### Auto-Send

Select whether blood glucose results are automatically sent to the paired pump after each test. Results sent using Auto-Send can be used by the pump system to calibrate a continuous glucose monitoring system or make insulin dosing calculations.

1




Turn the meter on by briefly pressing **OK**. From **Main Menu**, press ▼ to highlight **Settings**. Press **OK**.

2



Press ▼ to highlight **Wireless**. Press **OK**.

3

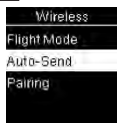

Press ▼ to highlight **Auto-Send**. Press **OK**.

4



Press ▼ or ▲ to highlight **On** or **Off**. Press **OK** to move ✓ to the option.

Press **OK** to set the option and return to the previous menu.

## 7 Wireless Communication and Meter Pairing

### WARNING

- Do not use alternative site testing (AST) to calibrate a continuous glucose monitoring system.
- Do not use alternative site testing to make insulin dosing calculations.
- Alternative site testing should be done only during steady-state times (when glucose is not changing rapidly).

### Data Transfer

This feature allows you to transfer data wirelessly from your meter to a paired pump. This feature can be used after the meter and paired pump have not been communicating for a period of time, such as when Flight Mode has been enabled or Auto-Send has been disabled. In this case, there may be blood glucose results that have not been sent to the pump.

Test results sent using Data Transfer cannot be used by the pump system to calibrate a continuous glucose monitoring system or to make insulin dosing calculations.

1




Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **My Data**. Press **OK**.

2



Press **▼** to highlight **Data Transfer**. Press **OK**.

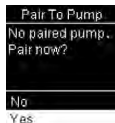


**If a pump is not paired with your meter:**


The meter prompts you to pair a pump to the meter (see the Pairing section in the chapter

Wireless Communication and Meter Pairing for details).

3

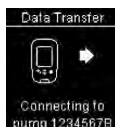

The meter transfers the data to the pump.

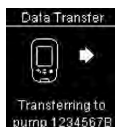


## 7 Wireless Communication and Meter Pairing

### Delete Pairing

This procedure is to delete the pairing on the meter, which ends communication between the meter and the pump. For instructions on deleting the pairing information on the pump, refer to the MiniMed™ 770G or MiniMed™ 780G System User Guide.

The meter can only be paired with 1 pump at a time. It is not necessary to delete the existing pairing when pairing with a new pump.

**1**

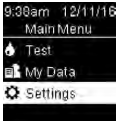

9:38am 12/11/16  
Main Menu  
Test  
My Data  
Settings

Turn the meter on by briefly pressing **OK**. From **Main Menu**, press **▼** to highlight **Settings**. Press **OK**.

**2**



Settings  
Time/Date  
Beeper  
Wireless  
More Options

Press **▼** to highlight **Wireless**. Press **OK**.


**3**



Wireless  
Flight Mode  
Auto-Send  
Pairing

Press **▼** to highlight **Pairing**. Press **OK**.


**4**



Pairing  
Pair To Pump  
Delete Pairing  
Help


Press **▼** or **▲** to highlight **Delete Pairing**. Press **OK**.

**5**



Delete Pairing  
pump 1234567B

Press **OK** to confirm.



Delete Pairing  
Deleted:  
pump 1234567B

The meter shows the deleted pairing.

## 8 Meter and Lancing Device Cleaning and Disinfecting

### What is the difference between cleaning and disinfecting?

Cleaning is the removal of dirt from the meter or lancing device.<sup>3</sup>

Disinfecting is the removal of most, but not all, disease-causing and other types of microorganisms (bloodborne pathogens) from the meter or lancing device.<sup>3</sup>

### Approved Cleaning and Disinfecting Product

The following product has been approved for cleaning and disinfecting the meter and lancing device:

#### **Super Sani-Cloth** (EPA\* reg. no. 9480-4)

Super Sani-Cloth can be purchased from Amazon.com, Officedepot.com, and Walmart.com.

- Do not use any other cleaning or disinfecting solutions. Using solutions other than the Super Sani-Cloth could result in damage to the meter and lancing device.
- The effect of using more than one product interchangeably to clean and disinfect the meter and lancing device has not been tested. Always use Super Sani-Cloth to clean and disinfect the meter and lancing device.
- Roche has demonstrated that the product is good for 5-year use, after testing in a total of 260 disinfection cycles (equal to cleaning and disinfecting once per week for 5 years).

### NOTE

For technical assistance or questions on cleaning and disinfecting, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

\*Environmental Protection Agency

## 8 Meter and Lancing Device Cleaning and Disinfecting

### Cleaning and Disinfecting the Meter

#### WARNING

If the meter is being operated by a second person who is providing testing assistance to the user, the meter and lancing device should be cleaned and disinfected prior to use by the second person.

To clean and disinfect without damaging the meter, follow these procedures carefully.

### When to Clean and Disinfect the Meter

- Clean the meter to remove visible dirt or other material prior to disinfecting.
- Clean and disinfect the meter at least once per week and when blood is present on the surface of the meter.
- Clean and disinfect the meter before allowing anyone else to handle the meter. Do not allow anyone else to use the meter on themselves for testing purposes.

#### NOTE

Using cleaning and disinfecting products could result in damage to the meter. If you notice any of the following signs of deterioration after cleaning and disinfecting your meter, stop using your meter and contact the Accu-Chek Customer Care Service Center at 1-800-858-8072: residue around buttons, clouding of display, button malfunction, out-of-range control results.

### What to Clean and Disinfect

The following parts of the meter should be cleaned and disinfected:

- The area around slots and openings (do not get any moisture in slots or openings)
- The meter display
- The entire meter surface

### How to Clean and Disinfect the Meter

#### WARNING

Failure to follow these instructions will damage the meter and stop it from working properly.

- DO NOT clean or disinfect the meter while performing a blood glucose or control test.
- DO NOT get any moisture in slots or openings.
- DO NOT spray anything onto the meter.
- DO NOT immerse the meter in liquid.

Always use the same product for both cleaning and disinfecting.




1



Wash hands thoroughly with soap and water.

## 8 Meter and Lancing Device Cleaning and Disinfecting

2

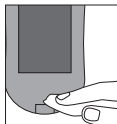

Turn the meter off and wipe the entire meter surface with a Super Sani-Cloth. Carefully wipe around the test strip slot and other openings.


Make sure that no liquid enters any slot or opening.

3

A separate Super Sani-Cloth should be used for cleaning and disinfection. For disinfecting the meter, get a new cloth and repeat step 2, making sure the surface stays wet for 2 minutes.

Make sure that no solution is seen in any slot or opening.

4

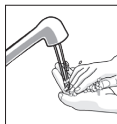

Wash hands thoroughly with soap and water.

### Cleaning and Disinfecting the Lancing Device

To clean and disinfect without damaging the lancing device, follow these procedures carefully.

### When to Clean and Disinfect the Lancing Device

- Clean the lancing device to remove visible dirt or other material prior to disinfecting.
- Clean and disinfect the lancing device at least once per week to remove visible dirt or other material for safe handling.
- Clean and disinfect the lancing device before allowing anyone else to handle the lancing device, for instance, if you have someone assisting you. Do not allow anyone else to use the lancing device.

### NOTE

- Do not throw away the cap after each use. Use the approved cleaning and disinfecting product on the cap.
- Always remove the lancet before cleaning or disinfecting the lancing device.
- Using cleaning and disinfecting products could result in damage to the lancing device. If you notice any of the following signs of deterioration after cleaning and disinfecting your lancing device, stop using your lancing device and contact the Accu-Chek Customer Care Service Center at 1-800-858-8072: residue around buttons, difficulty in priming the lancing device, difficulty in inserting the lancet.
- You might observe a slight discoloration of the lancing device after multiple cleaning and disinfecting cycles. This does not affect the functionality of the lancing device.



## 8 Meter and Lancing Device Cleaning and Disinfecting

### What to Clean and Disinfect

The following parts of the lancing device should be cleaned and disinfected:

- The entire lancing device surface
- The cap


### How to Clean and Disinfect the Lancing Device

#### WARNING

Failure to follow these instructions may damage the lancing device and stop it from working properly.


- DO NOT get any moisture into any openings.
- Always use the same product for both cleaning and disinfecting.

1

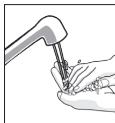

Wash hands thoroughly with soap and water.

2

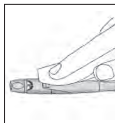

Wipe the entire surface of the lancing device and the inside of the cap with a Super Sani-Cloth.

3

A separate Super Sani-Cloth should be used for cleaning and disinfection. For disinfecting the lancing device, use a new cloth and repeat step 2 making sure the surface stays wet for 2 minutes.

4



Wash hands thoroughly with soap and water.

## 9 Meter Maintenance and Troubleshooting

### Meter Maintenance

The meter automatically tests its own systems every time you turn it on and lets you know if something is wrong. See the Error Messages section of this chapter.


If you drop the meter or think the results are not accurate, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

#### WARNING

Keep new and used batteries away from children. See the warning in the Important Safety Information section of this User's Manual for additional information.


### Changing the Batteries

1

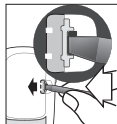


Open the child-resistant battery door by inserting a narrow object, such as a pen, into the slot. Push the tab in the direction of the arrow and lift the battery door up

2

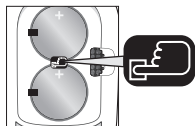

Release the old batteries by pressing the button. Remove the old batteries. Press and hold the Power/Set/OK button on the front of the meter for at least 2 seconds.

3

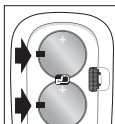

Slide the new batteries under the black tabs and the button, with the (+) side facing up. Put the battery door back in place and snap

it closed. Immediately discard the old batteries.

#### NOTE

- If the E-7 or E-9 error code still appears on the display after you have changed the batteries, remove the batteries again, press and hold the Power/Set/OK button for at least 2 seconds, then reinsert the batteries.
- Always have a spare set of batteries.
- Battery life may vary due to factors such as temperature and battery manufacturer.
- The meter uses two 3-volt lithium batteries, coin cell type CR2032. This type of battery can be found in many stores.
- Always replace both batteries at the same time and with the same brand.
- The logbook data is saved when you replace the batteries.

## 9 Meter Maintenance and Troubleshooting

### Error Messages

#### WARNING

- Never make therapy decisions based on an error message.
- If you have any concerns or see any other error message, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.



The meter will not turn on or the display is blank.

- Batteries are dead.

Insert new batteries.

- Display is damaged. / Meter is defective.

Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

- Extreme temperatures.

Move the meter to a location within the stated temperature limits.

⚠ Test Not Allowed  
Remove USB cable to perform test.

The meter is connected to a USB cable and a test cannot be performed.

Remove the USB cable and perform a test.

⚠ Transfer Failed  
Ensure pump is within range.

Blood glucose results were not transferred to the paired pump.

Make sure the paired pump is within range of the meter.

⚠ Transfer Not Allowed  
Wireless communication not allowed in flight mode.

Data cannot be sent to the paired pump because the meter is in Flight Mode.

Retry the data transfer when the meter is not in Flight Mode.

## 9 Meter Maintenance and Troubleshooting

⚠ No USB  
Features  
Remove USB  
cable.

USB functionality is not available for this meter.

Remove the USB cable.

⚠ Results  
Excluded  
One or more  
results may be  
excluded from  
averages.

One or more blood glucose results are excluded from the selected averages because the results are invalid or outside the system measuring range.

⚠ Results  
Excluded  
One or more  
results may be  
excluded from  
target % data.

One or more blood glucose results are excluded from the selected target % data because the test results are invalid.

⚠ Results  
Excluded  
One or more  
results may be  
excluded from  
low/high data.

One or more blood glucose results are excluded from the selected low/high data because the test results are invalid.

⚠ Invalid Date  
Check the date.

The date entered is not valid.

Enter the correct date.

⚠ Flight Mode  
Active  
Wireless is off.  
Selected setting  
was saved.

A meter setting was changed while in Flight Mode.


The setting change will not take effect until Flight Mode is turned off.

## 9 Meter Maintenance and Troubleshooting




Pairing to a pump cannot be performed while in Flight Mode.

Retry pairing when the meter is not in Flight Mode.

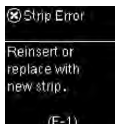


The test strip may be damaged, not properly inserted, or was previously used.

Remove and reinsert the test strip or replace it if damaged or previously used.

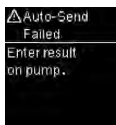

The blood glucose result has not been sent to the pump.


Manually enter the test result on the pump. For instructions, refer to the MiniMed™ 770G or MiniMed™ 780G System User Guide.

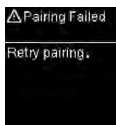

The meter was unable to pair with a pump.

Retry the pairing.

## 9 Meter Maintenance and Troubleshooting

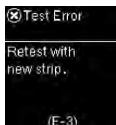

A meter or test strip error has occurred.

This error message could appear if the cap on the test strip container was not closed tightly. The test strips may have been damaged due to improper storage or handling.

**Never make therapy decisions based on an error message.**

Repeat the blood glucose test. If a second E-3 error message appears, perform a control test with the control solution and a new test strip. See the section Performing a Control Test in the chapter Control Tests. If you continue to receive an E-3 error message, use an alternate method for testing your blood glucose, such as a back-up meter and test strip. If the alternate method gives an extremely high blood glucose result, or if an alternate method is not available, **contact your healthcare professional immediately.**

In rare cases, an E-3 error message may indicate that your blood glucose is extremely high and above the system's measuring range. **See the Unusual Blood Glucose Results section in the chapter Blood Glucose Tests for other possible causes of the error message.**



Not enough blood or control solution was drawn into the test strip for measurement or was applied after the test had started.

Discard the test strip and repeat the blood glucose or control test.



Blood or control solution was applied to the test strip before **Apply drop** appeared.


Discard the test strip and repeat the blood glucose or control test.



An electronic error occurred.

Remove the batteries, press and hold the Power/Set/OK button for at least 2 seconds, and reinsert the batteries. Perform a blood glucose or control test.

## 9 Meter Maintenance and Troubleshooting

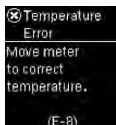


The temperature is above or below the proper range for the system.

Refer to the test strip package insert for system operating conditions. Move to an area with the appropriate conditions and repeat the blood glucose or control test. Do not artificially heat or cool the meter.




The batteries may be out of power.

Turn the meter back on. **If you are in a cold environment, move to a location with a more moderate temperature and retest.** If the message continues to appear after several attempts, replace the batteries. If the message reappears after the batteries have been replaced, remove the batteries, press and hold the Power/Set/OK button for at least 2 seconds, then reinsert the batteries.

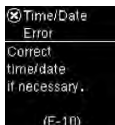

The time and date setting may be incorrect.

Make sure the time and date are correct and adjust, if necessary.

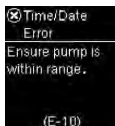

The time and date setting may be incorrect.

Make sure the pump is in communication range of the meter to update the time and date.



The test strip may be damaged.

Retest with a new test strip.

## 9 Meter Maintenance and Troubleshooting

⊗ High Ascorbate

Contact your  
healthcare  
professional.

(E-12)

Your blood sample may contain a high level of ascorbate.

Contact your healthcare professional.

⊗ Strip Error

Reinsert or  
replace with  
new strip.

(E-13)

Fluid or foreign material may be present in the test strip slot.

Remove and reinsert the test strip or repeat the blood glucose or control test with a new test strip. If the problem persists, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

⊗ Electronic  
Error

Contact Roche.

(E-14)

⊗

(E-14)

An electronic error has occurred.

Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

Sync Time

Meter time has  
been updated to  
match time on  
pump.

The meter time and date have been changed to match the paired pump.

Time/Date

Time and date  
cannot be  
changed while  
paired to a  
pump.

When paired to a pump, the meter time and date cannot be changed using the meter.

Logbook

No stored results

There are no results in the Logbook.

Logbook

mg/dL  
#100

Invalid result

There is an invalid result in the Logbook.



## 9 Meter Maintenance and Troubleshooting



There are no test results in range for the selected data.



Patterns is **On** but there are no active patterns stored in the meter.



Target Ranges is **Off** and there are no results for the Target % data stored in the meter.



Patterns is **Off**.



Target Ranges is **Off** and there are no results for the low/high data stored in the meter.



Blood glucose may be higher than the measuring range of the system.

See the Unusual Blood Glucose Results section in the chapter Blood Glucose Tests.



There are no results stored in the meter for the selected data.

## 9 Meter Maintenance and Troubleshooting

---



Blood glucose may be lower than the measuring range of the system.

See the Unusual Blood Glucose Results section in the chapter Blood Glucose Tests.



The batteries are almost out of power.

Change the batteries now. If the symbol reappears after the batteries have been replaced, remove the batteries again, press and hold the Power/Set/OK button for at least 2 seconds, then reinsert the batteries.

## Product Limitations

See the literature packaged with the test strips and control solutions for the latest information on product specifications and limitations.

Specifications	
Blood volume	Refer to the test strip package insert.
Sample type	
Measuring time	
Measuring range	
Test strip storage conditions	
System operating conditions	
Meter storage conditions	Temperature: -13–158 °F
Memory capacity	720 blood glucose results and 32 control results with time and date
Automatic off	90 seconds
Power supply	Two 3-volt lithium batteries (coin cell type CR2032)
Display	LCD
Dimensions	80 × 47 × 20 mm (LWH)
Weight	Approx. 40 g (with batteries)
Construction	Hand-held
Protection class	III
Meter type	The Accu-Chek Guide Link meter is suitable for continuous operation.

Control solution storage conditions	Refer to the control solution package insert.
Interfaces	Bluetooth low energy technology;  USB: micro-B connector (functionality not available)
Radio frequency connectivity	Bluetooth low energy technology operating in the frequency band of 2.402 GHz to 2.480 GHz with a maximum transmitted power of 0 dBm (1 mW).

**Bluetooth Wireless Technology** – The meter uses Bluetooth low energy wireless technology to communicate and transfer information to the pump. Bluetooth wireless technology is a form of radio frequency (RF) technology that operates in the unlicensed industrial, scientific, and medical band at 2.4 to 2.485 GHz.

**Electromagnetic Compatibility** – The meter meets the electromagnetic emission requirements as per EN ISO 15197. The chosen basis for electrostatic discharge immunity testing was basic standard IEC 61000-4-2. In addition, the meter meets the electromagnetic emissions requirements as per EN 61326-2-6 / EN 60601-1-2. Its electromagnetic emission is thus low. Interference from the meter to other electrically-driven equipment is not anticipated.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference, and

# 10 Technical Information

2. this device must accept any interference received, including interference that may cause undesired operation.

In the event there is interference from another device, it is recommended that you increase the distance between the meter and that device. You can also turn off the interfering device. In addition, you can turn off Bluetooth low energy wireless technology on the meter.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment is in direct contact with the body of the user under normal operating conditions. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.

Changes or modifications not expressly approved by the party responsible for compliance (i.e., the manufacturer) could void the user's authority to operate the equipment.

**Performance Analysis** – Refer to the test strip package insert.

**Test Principle** – Refer to the test strip package insert.

## Product Safety Information

### WARNING

- This meter meets IEC 61010-1 and IEC 61010-2-101 safety standards.
- Strong electromagnetic fields may interfere with the proper operation of the meter. Do not use the meter close to sources of strong electromagnetic radiation.
- To avoid electrostatic discharge, do not use the meter in a very dry environment, especially one in which synthetic materials are present.

## Travel Documentation

If you are traveling on a commercial airline, you may be required to provide documentation certifying that this meter meets environmental conditions and test procedures for Airborne Equipment (RTCA DO-160) section 21 Emission of Radio Frequency Energy. Visit [accu-chek.com](http://accu-chek.com) or contact the Accu-Chek Customer Care Service Center at 1-800-858-8072 to obtain a copy of the document.

## Discarding the Meter, Test Strips, Lancing Devices, Lancets, and Batteries

### WARNING

- Any product coming in contact with blood is considered contaminated (potentially infectious).\*
- During normal testing, any blood glucose meter may come in contact with blood.
- Lancing devices may also be considered sharps. Disposal of sharps is regulated by law in many jurisdictions.










Roche is committed to recycling and sustainability. Comply with any laws or ordinances relating to the disposal of sharps and/or contaminated products. Contact your local health department or other appropriate authorities for proper handling and disposal of used meters, used test strips, used lancets, and used batteries.












Consider the following points when discarding used testing materials: Consider recycling the meters and batteries at an appropriate facility. Be aware the meter is potentially hazardous electronics scrap (e-scrap) and should be disposed of accordingly. The batteries are potentially hazardous also and should be discarded accordingly.

Disinfect the meter before recycling or discarding.

\*29 CFR 1910.1030 – Bloodborne pathogens

## Explanation of Symbols

	Consult instructions for use or consult electronic instructions for use
	Caution, refer to safety-related notes in the instructions for use accompanying this product.
	Use by
	Sterilized using irradiation
	Use only once
	Single patient multiple use
	Wash your hands
	Biological risks
	3-volt coin cell type CR2032

	Keep new and used batteries away from children.
	Date of manufacture
	Medical device
	Manufacturer
 FCC ID: WX3-401	This device complies with Part 15 of the FCC Rules.
	Unique device identifier
	Catalogue number
	Serial number
	Batch code
	Global Trade Item Number
	Complies with the provisions of the applicable EU Legislation

## Additional Supplies

**Test Strips:** the Accu-Chek Guide test strips

**Control Solutions:** the Accu-Chek Guide control solutions

**Lancets:** the Accu-Chek Softclix 100-ct. lancets

## References

<sup>1</sup> FDA Public Health Notification: “Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens: Initial Communication, (2010). Update 11/29/2010.” <https://downloads.regulations.gov/FDA-2016-M-0035-0003/content.pdf>. Accessed February 23, 2022.

<sup>2</sup> CDC Clinical Reminder: “Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens, (2010).” [https://www.cdc.gov/injectionsafety/pdf/clinical\\_reminder\\_fingerstick\\_devices\\_riskbbp.pdf](https://www.cdc.gov/injectionsafety/pdf/clinical_reminder_fingerstick_devices_riskbbp.pdf). Accessed February 23, 2022.

<sup>3</sup> Centers for Disease Control and Prevention (CDC): “Guideline for Disinfection and Sterilization in healthcare Facilities, 2008.” Update: May 2019. William A. Rutala, Ph.D., M.P.H., and David J. Weber, M.D., M.P.H., and the Healthcare Infection Control Practices Advisory Committee (HICPAC). <https://www.cdc.gov/infectioncontrol/pdf/guidelines/disinfection-guidelines-H.pdf>. Accessed September 23, 2019.

# 11 License and Warranty

---

## Warranty

### **Accu-Chek Guide Link Meter 30-day Money-back Guarantee for Qualifying Consumers**

Roche offers qualifying consumers that purchase an Accu-Chek Guide Link meter, a 30-day money back guarantee. If you are not fully satisfied with your Accu-Chek Guide Link meter, contact the Accu-Chek Customer Care Service Center toll-free at 1-800-858-8072 to determine whether you qualify to receive a full refund within 30 days of purchase. If you are covered under Medicare, Medicaid, other federal/state programs, or private insurance you are NOT eligible for this money-back offer. Consumers affected by this exclusion may instead request a different Accu-Chek meter/system. The refund will be limited to the amount paid by you net of any rebates. You must have a copy of the dated itemized purchase receipt and the original packaging to obtain this refund.

### **Accu-Chek Guide Link Meter Limited 3-Year Warranty**

Roche warrants to the original purchaser of the meter that your Accu-Chek Guide Link meter will be free from defects in materials and workmanship for three years from the date of purchase. If, during this 3-year period, the meter does not work properly because of a defect in materials or workmanship, Roche will replace it with a new Accu-Chek Guide Link meter or equivalent product free of charge. The warranty on the replacement meter will expire on the date of the original warranty expiration or 90 days after the shipment of a replacement system, whichever period is longer. The purchaser's exclusive remedy with respect to the Accu-Chek Guide Link meter shall be replacement.

This warranty does not apply to the performance of an Accu-Chek Guide Link meter that has been damaged by accident or has been altered, misused, tampered with, or abused in any way. Roche will handle meters that show damage or abuse according to its Non-Warranty Service Policy described on the following page.

THE ABOVE WARRANTY IS EXCLUSIVE OF ALL OTHER WARRANTIES, AND ROCHE MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, THE IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE BE LIABLE TO THE PURCHASER OR ANY OTHER PERSON FOR ANY INCIDENTAL, CONSEQUENTIAL, INDIRECT, SPECIAL, OR PUNITIVE DAMAGES ARISING FROM OR IN ANY WAY CONNECTED WITH THE PURCHASE OR OPERATION OF THE METER OR ITS PARTS. NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IF ANY IS IMPLIED FROM THE SALE OF THE METER, SHALL EXTEND FOR A LONGER DURATION THAN THREE YEARS FROM THE DATE OF PURCHASE.

Some states do not allow limitations on how long an implied warranty will last or the exclusion of incidental or consequential damages, so the above limitation and exclusion may not apply to you. This warranty gives you specific legal rights, which vary from state to state.

# 11 License and Warranty

---

## **Non-Warranty Service Policy**

Roche Non-Warranty Service Policy applies to meters where the above warranty has not become effective, has become inapplicable, or has expired. Roche will replace, at its option, meters returned to it for a service charge (not to exceed \$35).

Replacement will be with the same or similar product. Replacement meters will be warranted for a period of 90 days from shipment under a limited warranty providing for replacement of parts and labor at no charge.

## **Warranty and Service Instructions**

All requests for return of Accu-Chek Guide Link meters under the above warranty or service policy must be made to the Accu-Chek Customer Care Service Center. You will be mailed a return authorization label, which must be affixed to your carton for shipping the system to Roche. Cartons received without this label will be returned to you at your expense.

Customers experiencing difficulties should review the troubleshooting information in the Meter Maintenance and Troubleshooting chapter of this manual. Further inquiries should be directed to the Accu-Chek Customer Care Service Center.

Be sure to register your warranty online at [accu-chek.com/register](http://accu-chek.com/register) for the Accu-Chek Guide Link system.



# 11 License and Warranty

## Limited License

**CAUTION – A RESTRICTED LICENSE LIMITS USE OF THE ACCU-CHEK GUIDE LINK SYSTEM (meter and test strips) IN THE UNITED STATES – READ CAREFULLY THE LIMITATIONS RECITED BELOW.**

The ACCU-CHEK Guide Link system (meter and test strips) and its use are protected by U.S. Patent Nos. 7,452,457 (expires 2-May-2026); 7,488,601 (expires 1-February-2026); 7,569,126 (expires 28-December-2026); 7,604,721 (expires 12-August-2026); 9,395,322 (exp. 13-March-2034); 10,119,931 (exp. 11-August-2035); 10,809,221 (exp. 13-March-2034); 9,594,052 (exp. 13-March-2034); 9,797,859 (exp. 29-October-2034); 10,295,494 (exp. 28-August-2035); 9,594,045 (exp. 13-March-2034). A license to use the ACCU-CHEK Guide Link system is required until the expiration of the last-to-expire patent listed above and is only granted when the ACCU-CHEK Guide Link meter is used with the ACCU-CHEK Guide test strips.

The ACCU-CHEK Guide test strips are specifically manufactured for operation with the ACCU-CHEK Guide Link meter. Use of other test strips supplied by another manufacturer may prevent or impair the proper function of the ACCU-CHEK Guide Link system.

Using the ACCU-CHEK Guide Link system indicates your acceptance of the restricted license to use the ACCU-CHEK Guide Link meter only with ACCU-CHEK Guide test strips. If you do not agree to the terms and conditions of the restricted license, you may return, at the place of purchase, the unused ACCU-CHEK Guide Link system for a full refund. If you have any questions, contact the ACCU-CHEK Customer Care Services Center at 1-800-858-8072.

Except where prohibited by statute, all warranties covering the ACCU-CHEK Guide Link system are voided by use of the ACCU-CHEK Guide Link meter with any test strips other than the ACCU-CHEK Guide test strips.

## Patent Information

U.S. Pat.: <http://www.roche-diagnostics.us/patents>

# Index

---

## A

alternative site testing 3, 20  
averages 39

## B

battery, changing 52  
battery type 61  
blood glucose results, unusual 21  
blood glucose test, performing 15  
blood glucose, warning 18  
button, functions 8

## C

comfort dial 7, 11  
comments, adding to test results 18  
control results, out-of-range 26  
control solution 23  
control test, performing 24

## D

data transfer, wireless 46

## H

high blood glucose 22  
hyperglycemia 22  
hypoglycemia 22

## L

lancing device, cleaning and disinfecting 50  
lancing device, using 11  
logbook 38  
low blood glucose 22  
low/high data 40

## M

maintenance, meter 52  
meter, cleaning and disinfecting 49

## P

patterns 42  
product limitations 61  
product safety information 62  
product specifications 61

## S

safety information 2  
settings, meter 27  
    beeper 29  
    language 37  
    last result 36  
    patterns 32  
    post meal 35  
    reminder 34  
    target ranges 30  
    time/date 29  
supplies 64  
symbols  
    blood glucose results 19  
    IVD 63  
    meter display 9  
symptoms, hypoglycemia/hyperglycemia 22

## T

target % data 39  
technical information 61  
test strips 13

## U

use by date 13, 23

## W

warranty 65  
wireless communication  
    auto-send 45  
    delete pairing 47  
    flight mode 44  
    overview 44  
    pairing 44





Roche Diabetes Care, Inc.  
9115 Hague Road  
Indianapolis, IN 46256, USA  
[www.accu-chek.com](http://www.accu-chek.com)

ACCU-CHEK, ACCU-CHEK GUIDE, and SOFTCLIX are trademarks of Roche.

MiniMed is a registered trademark of Medtronic, Inc.

The *Bluetooth*<sup>®</sup> word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Roche is under license.

All other product names and trademarks are the property of their respective owners.

© 2022 Roche Diabetes Care

09636129001B-1122

