ACCU-CHEK® Aviva Plus

Blood Glucose Monitoring System

**Owner's Booklet
for Single Patient Use Only**

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ACCU-CHEK®

Contents

Important Safety Information	3
Chapter 1: Your New System.....	7
Chapter 2: Control Tests.....	13
Chapter 3: The Accu-Chek Softclix Lancing Device	17
Chapter 4: Blood Glucose Tests.....	21
Chapter 5: Meter Memory, Setup, and Data Transfer.....	27
Chapter 6: Meter and Lancing Device Cleaning and Disinfecting.....	37
Chapter 7: Maintenance and Troubleshooting.....	43
Chapter 8: Technical Information.....	47
Index	55

Important Safety Information

The Accu-Chek Aviva Plus System

The Accu-Chek Aviva Plus blood glucose monitoring system is intended to be used for the quantitative measurement of glucose (sugar) in fresh capillary whole blood samples drawn from the fingertips, forearm, upper arm, or palm. The Accu-Chek Aviva Plus blood glucose monitoring system is intended to be used by a single person and should not be shared.

The Accu-Chek Aviva Plus blood glucose monitoring system is intended for self-testing outside the body (in vitro diagnostic use) by people with diabetes at home as an aid to monitor the effectiveness of diabetes control. The Accu-Chek Aviva Plus blood glucose monitoring system should not be used for the diagnosis of or screening of diabetes or for neonatal use. Alternate site testing should be done only during steady-state times (when glucose is not changing rapidly).

The Accu-Chek Aviva Plus test strips are for use with the Accu-Chek Aviva blood glucose meter to quantitatively measure glucose (sugar) in fresh capillary whole blood samples drawn from the fingertips, forearm, upper arm, or palm.

This system is intended to be used by a single person and should not be shared.

The Accu-Chek Aviva Plus blood glucose monitoring system includes:

Accu-Chek Aviva meter with battery, Accu-Chek Aviva Plus test strips*, Accu-Chek Aviva control solutions*, Accu-Chek Softclix lancing device* (with a black cap for fingertip testing and an AST cap** for alternate site testing), Accu-Chek Softclix lancets*.

*may be sold separately

**available upon request from the Accu-Chek Customer Care Service Center at 1-800-858-8072

WARNING

- Choking hazard. Small parts. Keep away from children under the age of 3 years.
- Keep new and used batteries away from children. Ingestion or insertion into the body may cause chemical burns, perforation of soft tissues, and death. Severe burns may occur within 2 hours of swallowing. If you think a battery might have been swallowed or placed inside any part of the body, seek medical attention immediately.
- If the battery compartment does not close securely, stop using the product and keep it away from children. Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

NOTE

The term "blood glucose" is used when referring to "blood sugar."

Important Safety Information

Need Help?

Contact the Accu-Chek Customer Care Service Center toll-free at 1-800-858-8072. Hours of operation are Monday through Friday between 8:00 am and 8:00 pm eastern standard time (EST). We offer assistance in many languages. You can also visit accu-chek.com for diabetes management tools and product demonstrations. Please register your warranty online at accu-chek.com/register, so you receive the best customer service possible and product update news.

About Testing Yourself or Others

WARNING

- DO NOT CHANGE YOUR THERAPY BASED ON A TEST RESULT THAT DOES NOT MATCH HOW YOU FEEL OR IF YOU BELIEVE THAT YOUR TEST RESULT COULD BE INCORRECT.
- It is always a good idea to have a back-up testing method available. Failure to test could cause a delay in therapy decisions and lead to a serious medical condition. Examples of back-up testing methods include a back-up meter or testing by a laboratory. Ask your healthcare professional or pharmacist about other possible back-up methods.
- If your blood glucose result does not match how you feel and you have followed the instructions in this Owner's Booklet, follow your healthcare professional's instructions, or contact your healthcare professional.

WARNING

- During normal testing, any blood glucose meter or lancing device may come in contact with blood. All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.^{1,2}
- The meter and lancing device should never be used by more than one person. Do not share the meter and lancing device with anyone, including family members, due to the risk of infection from bloodborne pathogens.^{1,2} Do not use on multiple patients!
- Cleaning and disinfecting the meter and lancing device destroys most, but not necessarily all, bloodborne pathogens.³
- If the meter is being operated by a second person who is providing testing assistance to the user, the meter and lancing device should be cleaned and disinfected prior to use by the second person.
- Disinfect the meter and lancing device before allowing anyone else to handle them. Do not allow anyone else to test with the meter or lancing device.
- It is important to keep the meter and lancing device clean and disinfected. For instructions on how to clean and disinfect the meter and lancing device, see Chapter 6, Meter and Lancing Device Cleaning and Disinfecting.
- Wash hands thoroughly before and after handling the meter, lancing device, or test strips.
- Failure to follow testing instructions or test strip storage and handling instructions can lead to an incorrect test result that may lead to improper therapy. Carefully read and follow the instructions in the User's Manual and package inserts for the test strips and control solutions.
- Inspect the test strip container before using the test strips for the first time. If you see any damage to the container, if anything prevents the cap from closing properly, or if the container was open before using for the first time, do not use the test strips. Do not perform a control test. Contact the Accu-Chek Customer Care Service Center. Damaged test strips can cause inaccurate results, which could lead to improper therapy.

Important Safety Information

- The meter, test strips, and control solution are only for use outside the body (in vitro). Do not eat the test strips. Do not swallow or inject the control solution or use the control solution for any purpose other than testing the Accu-Chek Aviva Plus system.

NOTE

- After inspecting the container per instructions above, perform a control test whenever opening a new test strip box. Performing a control test lets you know that the meter and test strips are working properly.
- Refer to the test strip and control solution package inserts for additional health-related information.

Special Information for Caregivers

- Do not use the meter system to measure blood glucose in people who are experiencing cardiovascular collapse (severe shock) or decreased peripheral blood flow.
- Consult your healthcare professional to determine if it is appropriate for your child to be taught how to use the meter system or any other medical products.
- Some people with diabetes do not experience symptoms of low blood glucose (hypoglycemia). Others, such as children or people who are unconscious or have certain disabilities, may not be able to communicate their symptoms to caregivers. For these reasons, do not change any therapy without first talking to a healthcare professional.
- Not for use on critically ill patients, patients in shock, dehydrated patients, or hyperosmolar patients.

Before You Start Testing

NOTE

Set the time and date on the meter before you begin testing.

The Accu-Chek Aviva Meter

1. **Power/Set Button** ①
Turns meter on or off and sets options.
2. **Display**
Shows results, messages, and test results stored in memory.
3. **Right Arrow and Left Arrow Buttons** ▶◀
Press to enter memory, adjust settings, and scroll through test results.
4. **Test Strip Slot**
Insert test strip here.

5. **Battery Door**
6. **Infrared (IR) Window**
Transfers data from meter to computer.

(for example)

**7. Test Strip Container*****8. Insert this end into meter.****9. Yellow Window**

Touch blood drop or control solution here.

10. Control Solution Bottle***11. Battery**

*Some items may not be included in the kit. They are a separate purchase.

The Accu-Chek Softclix Lancing Device

1. **Cap with Comfort Dial Depth Selection**
For fingertip testing.
2. **Ejector**
Slide to release lancet.
3. **Release Button**
Press to prick.
4. **Priming Button**
Press to prime.
5. **Lancet**
6. **AST Cap**
For Alternate Site Testing (available upon request from the Accu-Chek Customer Care Service Center at 1-800-858-8072).

NOTE

Some items may be sold separately.

Using the Accu-Chek Aviva Plus System

- Use only Accu-Chek Aviva Plus test strips.
- Use the test strip immediately after removing it from the test strip container.
- Do not apply blood or control solution to the test strip before inserting it into the meter. Retest with a new test strip. If a result appears before applying blood or control solution, do not act on that result.
- Close the test strip container tightly immediately after removing a test strip. Moisture can damage the test strips and produce incorrect results.
- Store the unused test strips in their original container with the cap tightly closed.
- Discard the test strips if they are past the **Use By** date printed on the test strip container. If the **Use By** date is missing or cannot be read, do not use the test strips. Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.
- Refer to the test strip package insert for test strip storage and system operating conditions.
- Do not remove test strips from the test strip container and put them into another container, such as a plastic bag, pocket, purse, wallet, etc.
- Do not reuse test strips. Once control solution or blood has been applied to a test strip, discard it. If a retest is necessary, use a new test strip.
- Perform a control test every time you open a new test strip box.

WARNING

To prevent inaccurate results:

- DO NOT expose test strips to heat, moisture, or humidity. Temperatures outside the required range, as well as moisture and humidity, can damage the test strips and lead to inaccurate results.
- DO NOT bend, cut, or alter the test strips.
- DO NOT get dirt, food, or other material on the test strip.

Button Functions

Here are the functions of the power/set and arrow buttons on the meter. These functions are used throughout this booklet. For more information on using these buttons during meter setup, see Chapter 5, Meter Memory, Setup, and Data Transfer.

Button	Function	Action
(①) (power/set button)	<p>Turn the meter on or off.</p> <p>Enter the set-up mode.</p> <p>Set the chosen feature.</p> <p>Exit the set-up mode at any time.</p> <p>Check the display segments.</p> 	<p>Press and release ①.</p> <p>Turn the meter on. Press and hold ① until set-up appears on the display.</p> <p>Press and release ①.</p> <p>Press and hold ① until the flashing test strip symbol appears on the display.</p> <p>Check the display before each test to make sure all segments are clear and look like the picture to the left. Turn the meter off. Press and hold ① to see the complete display.</p> <p>If one of the segments is missing or looks different from the picture, do not use the meter. Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.</p>
(◀ ▶) (right arrow and left arrow buttons)	<p>Adjust settings for time and date, beeper, test reminders, and hypoglycemic alert.</p> <p>Enter memory.</p> <p>Mark a blood glucose result.</p>	<p>Press and release ▲ or ▼.</p> <p>Press and hold ▲ or ▼ to scroll faster.</p> <p>Press and release ▲ or ▼.</p> <p>Press and release ▲ or ▼.</p>

When to Perform a Control Test

Performing a control test lets you know the meter and test strips are working properly. You should perform a control test when:

- you open a new test strip box.
- you think the test strips are damaged.
- you want to check the meter and test strips.
- the test strips were stored in extreme temperatures, humidity, or both.
- you dropped the meter.
- your test result does not match how you feel.
- you want to check if you are performing the test correctly.

About the Control Solution

- Use only Accu-Chek Aviva control solution.
- Close the control solution bottle tightly after use.
- Write the date you open the control solution bottle on the bottle label. The control solution must be discarded 3 months from the date the control solution bottle was opened (discard date) or on the **Use By** date on the bottle label, whichever comes first.
- Do not use control solution that is past the **Use By** or discard date.
- Refer to the control solution package insert for control solution storage conditions.
- The meter automatically recognizes the difference between the control solution and blood.
- The control results are not displayed in memory.
- The control solution can stain fabric. Remove stains by washing with soap and water.
- Control solution is available for purchase. To order the control solution, talk to your pharmacist or visit accu-cheek.com to order online.

2 Control Tests

Performing a Control Test

You need the meter, a test strip, and control solution Level 1 or Level 2. The control level is printed on the

1

bottle label.
Check the meter display
for missing segments.
All segments should look
like the picture.

2

Check the **Use By** date on
the test strip container.
Do not use test strips
past the **Use By** date.

3

or

Select the control solution
to test. You will enter the
level later in the test.

4

Insert the test strip into
the meter in the direction
of the arrows.
Place the meter on a flat
surface.

5

Remove the bottle cap.
Wipe the tip of the bottle
with a tissue.
Squeeze the bottle until a
tiny drop forms at the tip.

6

Touch the drop to the
front edge of the yellow
window of the test strip
until you see **L** flash. **Do not put control solution
on top of the test strip.**


7

(for example)

When you see **L** flash,
there is enough control
solution in the test strip.
The control result, the
bottle symbol, and a
flashing **L** appear on the
display. Do not remove
the test strip yet.

8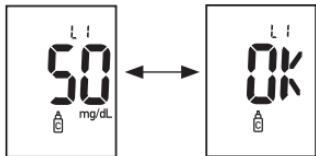

Wipe the tip of the bottle with a tissue. Cap the bottle tightly.

OK and the control result alternate on the display if the control result is in range OR **Err** and the control result alternate on the display if the control result is not in range.
Remove and discard the used test strip.

NOTE

- Most people just test the Level 1 control. If you wish, you can also test a Level 2 control. A set of Level 1 and Level 2 control solutions is available for purchase. To order control solutions, talk to your pharmacist or visit accu-chek.com to order online.
- The ranges for Level 1 and Level 2 control solutions are printed on the test strip container label.
- After a successful test, the meter turns off 5 seconds after the test strip is removed.

2 Control Tests

Understanding Out-of-Range Control Results

WARNING

- The control range applies only to a control result. It only indicates that the test strips and meter are working properly. Do not use a control result to interpret blood glucose results.
- If the control result is out of range, do not use the meter until you solve the problem. Check this list to help solve the problem.

Troubleshooting Checks	Action
1. Were the test strips or control solutions expired?	Discard the test strips or control solution if either is past the Use By date. If the control solution was opened more than 3 months ago, discard it. Repeat the control test with an unexpired test strip and an unexpired control solution.
2. Did you wipe the tip of the control solution bottle before use?	Wipe the tip of the bottle with a tissue. Repeat the control test with a new test strip and a fresh drop of control solution.
3. Were the caps on the test strip container and the control solution bottle always closed tightly?	Replace the test strips or control solution if you think either was uncapped for some time. Repeat the control test.
4. Was the test strip used immediately after it was removed from the test strip container?	Repeat the control test with a new test strip and a fresh drop of control solution.
5. Were the test strips and control solutions stored in a cool, dry place?	Repeat the control test with a properly stored test strip or control solution.
6. Did you follow the directions?	Read Chapter 2, Control Tests, and repeat the control test.
7. Did you choose the correct control solution level, either 1 or 2, when you performed the test?	If you chose the wrong control solution level, you can still compare the control result to the range printed on the test strip container.
8. Are you still unsure of the problem?	Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

Using the Accu-Chek Softclix Lancing Device

- The black cap is for fingertip testing only.
- Remove the cap by pulling it straight off. Do not twist the cap.

- The AST cap* is for testing other approved sites on the body, not the fingertip.
- Remove the cap by pulling it straight off. Do not twist the cap.


- The penetration depth indicator on the comfort dial shows the current depth setting. The higher the number, the deeper the penetration. The best depth setting is the lowest number that lets you get enough blood for a test. Try different depth settings to find the one that is right for you.
- For soft skin, we suggest a depth setting of 2. For thick skin, try a higher depth setting.
- We suggest starting at a depth setting of 5½ for forearm or upper arm testing, and a depth setting of 3 for palm testing. Once you perform a successful blood glucose test, you may want to find the lowest depth setting that provides enough blood with the least amount of pain.

*Available upon request from the Accu-Chek Customer Care Service Center at 1-800-858-8072.


3 The Accu-Chek Softclix Lancing Device

Inserting a Lancet


You must first insert a lancet into the lancing device to get it ready for use.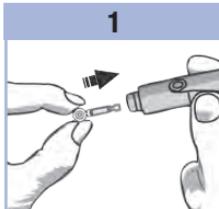

Remove the cap.
Insert a new lancet until it
clicks firmly into place.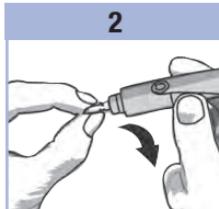

Twist off the lancet's
protective cover.

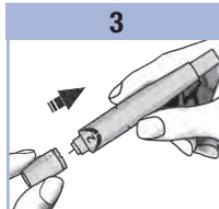Slide the cap on until it
stops by aligning the
notch on the cap with the
notch on the lancing
device.

Using the Accu-Chek Softclix Lancing Device

1

Make sure you select the correct cap.

2

Adjust the lancet depth by turning the comfort dial.

3

Press the priming button in as far as it will go, like a pen.
The release button turns yellow when the device is ready.

4

Obtain a blood drop. See Chapter 4, Blood Glucose Tests.


NOTE

- Do NOT press the release button while pressing the priming button.
- ALWAYS use a new, sterile lancet each time you test to avoid infection.
- NEVER reuse a lancet.
- NEVER share your lancing device with anyone.

3 The Accu-Chek Softclix Lancing Device

WARNING

- The meter and lancing device should never be used by more than one person. Do not share the meter and lancing device with anyone, including family members, due to the risk of infection from bloodborne pathogens.^{1,2} Do not use on multiple patients!
- Blood glucose results can be displayed in either mg/dL or mmol/L. The back label of the meter shows the unit of measurement. If the meter shows the wrong unit, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072. If you do not know which unit of measurement is correct for you, contact your healthcare professional. Using the wrong unit of measurement may cause misinterpretation of your actual blood glucose level and may lead to improper therapy.

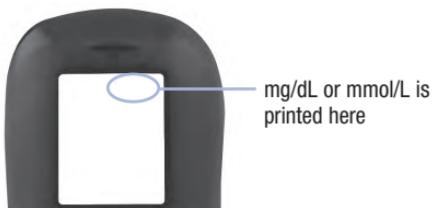**Performing a Blood Glucose Test with Blood from Your Fingertip**

Refer to the Important Safety Information before performing a blood glucose test.

Before you perform your first blood glucose test, set up the meter correctly and perform a control test. You need the meter, a test strip, and a lancing device with a lancet inserted to perform a blood glucose test.

1

Check the meter display for missing segments. All segments should look like the picture.

2

Wash your hands with warm soapy water and dry thoroughly. Dirty or wet hands could affect test results.

3

Adjust the comfort dial.

4

Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.


5

Insert the test strip into the meter in the direction of the arrows.

The meter turns on and beeps.

6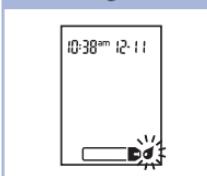

When the blood drop symbol flashes, obtain a blood drop.

7

Place the lancing device firmly against the side of your fingertip.

Press the yellow release button to prick your finger.

8

Gently squeeze your finger to assist the blood flow.

This helps you get a blood drop.

9

Touch the blood drop to the **front edge** of the yellow window of the test strip.

Do not put blood on top of the test strip.

The meter beeps and flashes when there is enough blood in the test strip.

10

The test result appears on the display.

To assign a marker to the test result, leave the test strip in the meter. See Chapter 4, Marking Blood Glucose Results and Setting the Post-Meal Test Reminder.

11

Otherwise, remove and discard the used test strip. After a successful test, the meter turns itself off 5 seconds after the test strip is removed. Discard the used lancet. Remove the cap and point the lancet away from you. Slide out the ejector to release the lancet into a puncture-proof container. **Always discard according to local regulations.**

12

Wash hands thoroughly with soap and water.

Performing a Blood Glucose Test with Blood from Your Palm, Forearm, or Upper Arm (Alternate Site Testing)

WARNING

Do not use Alternate Site Testing to calibrate a continuous glucose monitoring system or to make insulin dosing calculations.

You have the option of obtaining a blood sample from other sites on your body besides the fingertip. Alternate sites include the palm, forearm, and upper arm.

Blood obtained from the fingertip can be used at any time to test blood glucose.

If blood from an alternate site is used, there are certain times when testing is not appropriate. This is because your blood glucose level changes faster in your fingertip than in the alternate sites. These differences may cause you to misinterpret your actual blood glucose level, leading to improper therapy and potential adverse health effects.

Read the next section before you try testing from alternate sites.

You may perform a palm, forearm, or upper arm test	<ul style="list-style-type: none">• immediately before a meal.• while fasting.
You may NOT perform a palm, forearm, or upper arm test	<ul style="list-style-type: none">• up to 2 hours following a meal, when blood glucose values can rise quickly.• after injecting bolus insulin, when blood glucose values can decrease rapidly.• after exercise.• if you are sick.• if you think your blood glucose is low (hypoglycemia).• if you sometimes do not notice when your blood glucose is low.

If you are interested in AST, talk to your healthcare professional first.

To obtain an AST cap and detailed AST instructions, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

4 Blood Glucose Tests

Marking Blood Glucose Results and Setting the Post-Meal Test Reminder

You can mark a blood glucose result to indicate a special event. If you select a test result marker, it is automatically stored in memory. When you review the test results in memory, these markers can help you remember what was different about the test result.

Symbol	Function
	Pre-Meal Marker Marking blood glucose results with a pre-meal marker provides more information about your test results to help you and your healthcare professional in the management of your diabetes.
	Pre-Meal Marker with Post-Meal Test Reminder When a pre-meal result is marked with a post-meal test reminder, the meter beeps 1 or 2 hours after you test to remind you to do a post-meal test.
	Post-Meal Marker Marking blood glucose results with a post-meal marker provides more information about your test results to help you and your healthcare professional in the management of your diabetes. The post-meal marker automatically appears with the test result if a test is performed 15 minutes before or after the post-meal test reminder is programmed to beep. Set the post-meal test reminder time to 1 or 2 hours in the set-up mode. The meter will beep 1 or 2 hours after the pre-meal test to remind you to do a post-meal test.
	General Marker You might want to use the general marker to mark an event such as an AST result or exercise.

Here is how to mark a test result and initiate a post-meal test reminder:

1. Perform a blood glucose test. The blood glucose result appears on the display.
2. Leave the test strip in the meter. Press and release or to toggle through the test result markers and post-meal test reminder.
3. When the marker or test reminder that you wish to select appears on the display, remove the test strip from the meter.

Unusual Blood Glucose Results

If your blood glucose result does not match how you feel, follow these steps:

1. Perform a control test. See Chapter 2, Control Tests.
2. Repeat the blood glucose test. See Chapter 4, Performing a Blood Glucose Test with Blood from Your Fingertip.

If your blood glucose result still does not match how you feel, follow your healthcare professional's instructions or call your healthcare professional immediately.

NOTE

Always follow your healthcare professional's instructions. For example, if your healthcare professional has advised you to immediately treat a low blood glucose result (such as by eating something), then do that first.

WARNING

- Do not change your treatment because of one blood glucose result.
- NEVER ignore symptoms of low or high blood glucose.

Symptoms of Low or High Blood Glucose

WARNING

The meter is designed to provide a numerical value for blood glucose in the range of 20–600 mg/dL. If you receive a numerical value below 20 mg/dL or above 600 mg/dL and it does not match how you feel, contact the Accu-Chek Customer Care Service Center toll-free at 1-800-858-8072.

Being aware of the symptoms of low or high blood glucose can help you understand your test results and decide what to do if they seem unusual.

Low blood glucose (hypoglycemia): Symptoms of hypoglycemia may include, but are not limited to, anxiety, shakiness, sweating, headache, increased hunger, dizziness, pale skin color, sudden change in mood or irritability, fatigue, difficulty concentrating, clumsiness, palpitations, and/or confusion.

High blood glucose (hyperglycemia): Symptoms of hyperglycemia may include, but are not limited to, increased thirst, frequent urination, blurred vision, drowsiness, and/or unexplained weight loss.

If you are experiencing any of these symptoms, or other unusual symptoms, test your blood glucose from the fingertip. If your blood glucose result is displayed as LO or HI, follow your healthcare professional's instructions or contact your healthcare professional immediately. If your blood glucose result does not match how you feel, follow the steps under Chapter 4, Unusual Blood Glucose Results.

Comparing Your Meter Result to a Laboratory Result

A common question is how the blood glucose results on the meter compare to the laboratory results. Your blood glucose can change quickly, especially after eating, taking medication, or physical activity. If you test yourself in the morning, then go to your healthcare professional's office for a blood glucose test, your test results will probably not match, even if you are fasting. This is typically not a problem with the meter, it just means that time has elapsed and your blood glucose level has changed.

Although you always apply fresh capillary whole blood to the test strip, the system has been calibrated to deliver plasma-like values for easier comparison to laboratory results.

If you want to compare your meter result to the laboratory result, **you must be fasting**. Take the meter to your healthcare professional's office and test yourself by fingerstick within 5 minutes of having blood drawn from your arm by a healthcare professional. Keep in mind that the laboratory uses different technology than the meter and that blood glucose meters for self-testing generally read somewhat lower than the laboratory result.

Memory

Storing Blood Glucose and Control Results

The meter automatically stores up to 500 blood glucose results with the time and date of the test result and any test result marker which you can review at any time. Test results are stored from the newest to the oldest, so set the time and date correctly in the meter. Having the correct time and date setting helps ensure appropriate interpretation of stored blood glucose results by you and your healthcare team.

NOTE

- The memory is not lost when the battery is replaced; however, confirm that the time and date are still correct. See Chapter 5, Setting the Time and Date.
- Once 500 blood glucose results or 20 control results are in memory, adding a new result causes the oldest one to be deleted.
- If more than 500 blood glucose tests are performed within a 90-day period, only the most recent 500 test results are included in the 90-day average.
- The control results are stored in memory, but cannot be reviewed on the meter. To view stored control results, first transfer them to a compatible software application. For product availability, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.
- Control results are not included in the 7-, 14-, 30-, and 90-day averages.
- Only the test results that have been assigned a pre-meal or post-meal marker are included in pre-meal and post-meal averages. All blood glucose results are included in the general 7-, 14-, 30-, and 90-day averages.

5 Meter Memory, Setup, and Data Transfer

Memory

Viewing Test Results in Memory

With the meter on or off, press and release or to enter memory. The most recent test result appears on the display.

Button	Function	Action
 (left arrow button)	Previous test results	Press to view previous test results from newest to oldest. ➡ ➡
 (right arrow button)	General averages	Press to view 7-, 14-, 30-, and 90-day averages. n = number of test results in the average ➡ ➡
 (right arrow button)	Pre-meal averages Post-meal averages	Continue to press to view 7-, 14-, 30-, and 90-day pre-meal and post-meal averages. Only test results with a pre-meal or post-meal marker are included in the pre-meal or post-meal averages.

WARNING

Do not change your therapy based on an individual test result in memory. Talk to your healthcare professional before changing therapy based on test results in memory.

Setup**Using the Set-Up Mode**

The following features can be customized as required.

Symbol	Function	Action	Factory Pre-set
	Time and date	Set the time and date.	
	Beeper	Select On or OFF . Setting the beeper to OFF does not affect test results. The beeper must be On to use the test reminders feature.	
	Post-meal test reminder	Select 1 hour or 2 hours. The meter will beep 1 or 2 hours after the pre-meal test to remind you to do a post-meal test.	
	Test reminders	Select On or OFF . You can set up to 4 test reminders per day.	A-1 8:00 am A-2 12:00 pm (noon) A-3 6:00 pm A-4 10:00 pm
	Hypoglycemic alert	Select On or OFF . The hypoglycemic alert can be set to a level between 50 and 90 mg/dL to let you know when your blood glucose is possibly too low.	

5 Meter Memory, Setup, and Data Transfer

Setup

Setting the Time and Date

1

Press and release ① (power/set button) to turn the meter on.
The flashing test strip symbol appears on the display.

2


Press and hold ① until set-up appears on the display.
The hour flashes.

3

Press and release ▶ or ◀ to decrease or increase the hour.
Press and release ① to set the hour.
The minutes flash.

4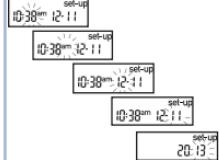

Repeat step 3 to set minutes, am/pm, month, day, and year.
To set up more options, press and release ①.
To exit, press and hold ① until the flashing test strip symbol appears on the display.

Setup

Setting the Beeper On or OFF

NOTE

The beeper prompts you:

- to apply blood or control solution to the test strip.
- when enough blood or control solution is drawn into the test strip.
- when the blood glucose or control test is complete.
- when a button is pressed.
- when it is time to perform a test (if you set the test reminders or post-meal test reminder).
- if an error occurred while performing a blood glucose or control test (even if the beeper is off, it still beeps for an error).

1


Press and release ① to turn the meter on.

The flashing test strip symbol appears on the display.

Press and hold ① until set-up appears on the display.

2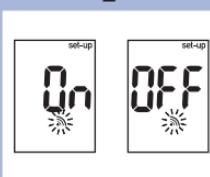

Press and release ① repeatedly until the flashing beeper symbol and On or OFF appear on the display.

3

Press and release ▶ or ▷ to switch between On and OFF.

To set up more options, press and release ①.

To exit, press and hold ① until the flashing test strip symbol appears on the display.

5 Meter Memory, Setup, and Data Transfer

Setup


Setting the Post-Meal Test Reminder

NOTE

The post-meal test reminder:

- beeps 1 or 2 hours after the blood glucose test to remind you to do a post-meal test.
- beeps every 2 minutes up to 3 times.
- turns off by inserting a test strip or pressing any button.


- If a test was performed within 15 minutes of a test reminder, the test reminder does not beep.
- If the meter is on at the test reminder time, the test reminder does not beep.
- Exposure to cold conditions may disable test reminders until the meter is turned on.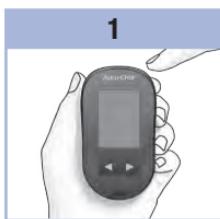

Press and release ① to turn the meter on.

The flashing test strip symbol appears on the display.

Press and hold ① until set-up appears on the display.

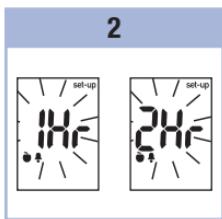Press and release ① repeatedly until set-up and the flashing 1Hr or 2Hr appear on the display.

3

Press and release ▶ or ▷ to select 1Hr or 2Hr.

Press and release ① to set the option.

To set up more options, press and release ①.

To exit, press and hold ① until the flashing test strip symbol appears on the display.

This display appears when a post-meal test reminder occurs.

Setup


Setting the Test Reminders

NOTE


Test reminders:

- beep at the same time each day.
- beep every 2 minutes up to 3 times.
- turn off by inserting a test strip or pressing any button.


- If a test was performed within 15 minutes of a test reminder, the test reminder does not beep.
- If the meter is on at the test reminder time, the test reminder does not beep.
- Exposure to cold conditions may disable test reminders until the meter is turned on.
- If a test reminder is off in set-up mode, any subsequent test reminders are also off. For example, if you set A-1 but turn off A-2, then A-3 and A-4 will automatically be off.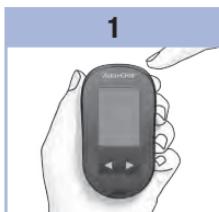


Press and release ① to turn the meter on.
The flashing test strip symbol appears on the display.
Press and hold ① until **set-up** appears on the display.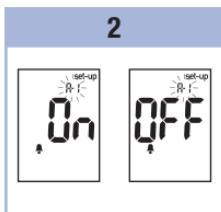

Press and release ① repeatedly until the bell symbol, **OFF**, **set-up**, and the flashing **A-1** appear on the display.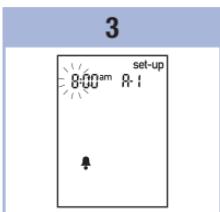

Press and release ② or ③ to switch between **On** and **OFF**.
Press and release ① to set your choice.
If **On** is selected, the hour flashes.

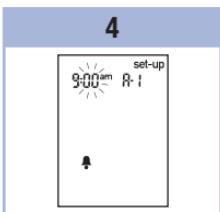Press and release ② or ③ to adjust the hour.
Press and release ① to set the hour.
The minutes flash.

5 Meter Memory, Setup, and Data Transfer

5

Press and release or to select **00, 15, 30, or 45**. These are the only choices.
Press and release to set the minutes.
The **am** or **pm** flashes.

6

Press and release or to switch between **am** or **pm**.
Press and release to set **am** or **pm**.
The bell symbol, **OFF**, and the flashing **A-2** appear on the display.

7

Either set the **A-2** test reminder or press and release to set up more options.
To exit, press and **hold** until the flashing test strip symbol appears on the display.

Setup

Setting the Hypoglycemic Alert

WARNING

- This function is no substitute for hypoglycemia training by your healthcare professional.
- Before you set the hypoglycemic alert, talk to your healthcare professional to help you decide what blood glucose level is your hypoglycemic level.

1


Press and release ① to turn the meter on.

The flashing test strip symbol appears on the display.

Press and **hold** ① until **set-up** appears on the display.

2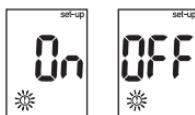

Press and release ① repeatedly until **set-up**, **OFF**, and the flashing ① appear on the display.

3

Press and release ▶ or ▷ to switch between **On** and **OFF**.

Press and release ① to set your choice.

If you select **On**, **set-up** appears and ① flashes.

4

Press and release ▶ or ▷ to adjust the level.

Press and release ① to set the level.

To exit, press and **hold** ① until the flashing test strip symbol appears on the display.

The set-up mode is complete.

Data Transfer

Transferring Results to a Computer

We offer a variety of software to help you transfer your results. For information on Accu-Chek software, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

You can transfer your stored results to a computer to track, identify patterns, and print.

Transferring Data Directly to a Computer Using Specialized Software and an Infrared Cable

1. Install the software according to the instructions.
2. To transfer the results to a computer, connect the infrared cable according to the instructions.
3. Run the software program and follow the instructions for data transfer. Make sure the software is ready to accept data from the meter.
4. With the meter off, press and **hold** both and until 2 arrows on the display alternately flash.
5. Locate the infrared (IR) window on the top of the meter.
6. Locate the IR window on the infrared cable.
7. Place the meter on a flat surface. Point the 2 IR windows toward each other. They should be 1–4 inches apart.
8. Do not move the infrared cable or meter during the data transfer.
9. Follow the prompts on the software.
10. The software program may shut off the meter automatically when the data transfer is complete. Should this occur, follow the prompts on the computer screen.

NOTE

- If the data did not transfer successfully, try again. Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072 if you still have problems.
- To make the most of the transfer feature, make sure the meter is set to the correct time and date.

What is the difference between cleaning and disinfecting?

Cleaning is the removal of dirt from the meter or lancing device.³

Disinfecting is the removal of most, but not all, disease-causing and other types of microorganisms (bloodborne pathogens) from the meter or lancing device.³

Approved Cleaning and Disinfecting Product

The following product has been approved for cleaning and disinfecting the meter and lancing device:

Super Sani-Cloth (EPA* reg. no. 9480-4) *Environmental Protection Agency

Super Sani-Cloth can be purchased from Amazon.com, OfficeDepot.com, and Walmart.com.

- Do not use any other cleaning or disinfecting solutions. Using solutions other than the Super Sani-Cloth could result in damage to the meter and lancing device.
- The effect of using more than one product interchangeably to clean and disinfect the meter and lancing device has not been tested. Always use Super Sani-Cloth to clean and disinfect the meter and lancing device.
- Roche has tested the approved product for a total of 260 cycles, which is equal to cleaning and disinfecting once per week for a 5-year period.

NOTE

For technical assistance or questions on cleaning and disinfecting, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

6 Meter and Lancing Device Cleaning and Disinfecting

Cleaning and Disinfecting the Meter

WARNING

If the meter is being operated by a second person who is providing testing assistance to the user, the meter and lancing device should be cleaned and disinfected prior to use by the second person.

To clean and disinfect without damaging the meter, follow these procedures carefully.

When to Clean and Disinfect the Meter

- Clean the meter to remove visible dirt or other material prior to disinfecting.
- Clean and disinfect the meter at least once per week and when blood is present on the surface of the meter.
- Clean and disinfect the meter before allowing anyone else to handle the meter. Do not allow anyone else to use the meter on themselves for testing purposes.

NOTE

Using cleaning and disinfecting products could result in damage to the meter. If you notice any of the following signs of deterioration after cleaning and disinfecting your meter, stop using your meter and contact the Accu-Chek Customer Care Service Center at 1-800-858-8072: residue around buttons, clouding of display, button malfunction, out-of-range control results.

What to Clean and Disinfect

The following parts of the meter should be cleaned and disinfected:

- The area around slots and openings (do not get any moisture in slots or openings)
- The meter display
- The entire meter surface

How to Clean and Disinfect the Meter

WARNING

Failure to follow these instructions will damage the meter and stop it from working properly.

- DO NOT clean or disinfect the meter while performing a blood glucose or control test.
- DO NOT get any moisture in slots or openings.
- DO NOT spray anything onto the meter.
- DO NOT immerse the meter in liquid.
- Always use the same product for both cleaning and disinfecting.

1

Wash hands thoroughly with soap and water.

2

Turn the meter off and wipe the entire meter surface with a Super Sani-Cloth. Carefully wipe around the test strip slot and other openings.
Make sure that no liquid enters any slot or opening.

3

A separate Super Sani-Cloth should be used for cleaning and disinfection. For disinfecting the meter, get a new cloth and repeat step 2, making sure that the surface stays wet for 2 minutes.

Make sure that no solution is seen in any slot or opening.

4

Wash hands thoroughly with soap and water.

6 Meter and Lancing Device Cleaning and Disinfecting

Cleaning and Disinfecting the Lancing Device

To clean and disinfect without damaging the lancing device, follow these procedures carefully.

When to Clean and Disinfect the Lancing Device

- Clean the lancing device to remove visible dirt or other material prior to disinfecting.
- Clean and disinfect the lancing device at least once per week to remove visible dirt or other material for safe handling.
- Clean and disinfect the lancing device before allowing anyone else to handle the lancing device, for instance, if you have someone assisting you. Do not allow anyone else to use the lancing device.

NOTE

- Do not throw away the cap after each use. Use the approved cleaning and disinfecting product on the cap.
- Always remove the lancet before cleaning or disinfecting the lancing device.
- Using cleaning and disinfecting products could result in damage to the lancing device. If you notice any of the following signs of deterioration after cleaning and disinfecting your lancing device, stop using your lancing device and contact the Accu-Chek Customer Care Service Center at 1-800-858-8072: residue around buttons, difficulty in priming the lancing device, difficulty in inserting the lancet.
- You might observe a slight discoloration of the lancing device after multiple cleaning and disinfecting cycles. This does not affect the functionality of the lancing device.

What to Clean and Disinfect

The following parts of the lancing device should be cleaned and disinfected:


- The entire lancing device surface
- The cap

How to Clean and Disinfect the Lancing Device


WARNING

Failure to follow these instructions may damage the lancing device and stop it from working properly.


- DO NOT get any moisture into any openings.
- Always use the same product for both cleaning and disinfecting.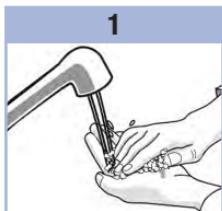

1
Wash hands thoroughly with soap and water.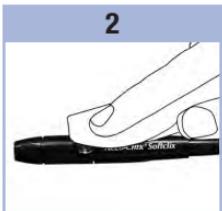

2
Wipe the entire surface of the lancing device and the inside of the cap with a Super Sani-Cloth.

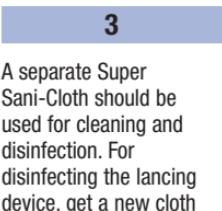3
A separate Super Sani-Cloth should be used for cleaning and disinfection. For disinfecting the lancing device, get a new cloth and repeat step 2, making sure that the surface stays wet for 2 minutes.

4
Wash hands thoroughly with soap and water.

6 Meter and Lancing Device Cleaning and Disinfecting

Meter Maintenance

The meter automatically tests its own systems every time you turn it on and lets you know if something is wrong. See Chapter 7, Display and Error Messages.

If you have problems with the meter or think the results are not accurate, perform a control test with an unexpired test strip and control solution. If the control result is not within the acceptable range, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

WARNING

Keep new and used batteries away from children. See the warning in the Introduction of this User's Manual for additional information.


Changing the Battery

Push the tab in the direction of the arrow to open the battery door on the back of the meter.

Remove the old battery and place the new one in the meter with the (+) side facing up.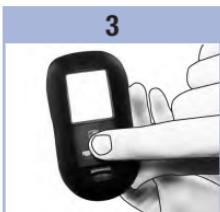

Put the battery door back into position and snap it into place.
The meter prompts you for a time and date confirmation when it is turned on.

NOTE



- The meter uses one 3-volt lithium battery, coin cell type CR2032. This type of battery can be found in many stores. It is a good idea to have a spare battery available.
- Be sure the battery is inserted **(+) side facing up**.

Display and Error Messages

WARNING

Never make therapy decisions based on an error message.

If you have any concerns or see any other error display, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

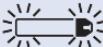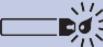Display	Action
The meter will not turn on or the display is blank.	<ul style="list-style-type: none">Battery is dead. Insert new battery.Display is damaged. Contact the Accu-Chek Customer Care Service Center.Meter is defective. Contact the Accu-Chek Customer Care Service Center.Extreme temperatures. Move the meter to a more temperate area.
	Battery power is low. Change the battery soon.
	The meter is in set-up mode, waiting for you to change or confirm settings.
	The meter is ready for you to insert a test strip.
	The meter is ready for a drop of blood or control solution.
	Blood glucose may be higher than the measurement range of the system. See Chapter 4, Unusual Blood Glucose Results.
	Blood glucose may be lower than the measurement range of the system. See Chapter 4, Unusual Blood Glucose Results.

Display	Action
	Blood glucose is below the defined hypoglycemic (low blood glucose) level. See Chapter 4, Unusual Blood Glucose Results.
	A general marker was assigned to this test result.
	A pre-meal marker was assigned to this test result.
	A post-meal marker was assigned to this test result.
	A pre-meal marker was assigned to this test result and the post-meal test reminder has been activated.
E-1	The test strip may be damaged or not properly inserted. Remove and reinsert the test strip, or replace it if damaged.
E-3	Your blood glucose may be extremely high or a meter or a test strip error has occurred. <ul style="list-style-type: none">• If your test result matches how you feel, contact your healthcare professional immediately.• If your test result does not match how you feel, repeat the blood glucose test. See Chapter 4, Unusual Blood Glucose Results.<ul style="list-style-type: none">• If the E-3 code still appears for your blood glucose test, your blood glucose result may be extremely high and above the system's reading range. Contact your healthcare professional immediately.• If the second test result does not match how you feel, perform a control test with the control solution and a new test strip.<ul style="list-style-type: none">• If the control result is within the acceptable range, review the proper testing procedure and repeat the blood glucose test with a new test strip.• If the control result is not within the acceptable range, see Chapter 2, Understanding Out-of-Range Control Results.

Display	Action
E-4	Not enough blood or control solution was drawn into the test strip for measurement or was applied after the test had started. Discard the test strip and repeat the blood glucose or control test.
E-6	Blood or control solution was applied to the test strip before the flashing drop symbol appeared on the display. Discard the test strip and repeat the blood glucose or control test.
E-7	An electronic error occurred, or in rare cases, a used test strip was removed and reinserted. Turn the meter off and on, or take the battery out for 20 seconds and reinsert it. Perform a blood glucose or control test.
E-8	The temperature is above or below the proper range for the system. Refer to the test strip package insert for system operating conditions. Move to an area with the appropriate conditions, wait 5 minutes, and repeat the blood glucose or control test. Do not artificially heat or cool the meter.
E-9	The battery is almost out of power. Change the battery now. If the message reappears after the battery has been replaced, remove the battery again, press any meter button, then reinsert the battery.
E-10	The time and date settings may be incorrect. Make sure the time and date are correct and adjust, if necessary.

Product Limitations

See the literature packaged with the test strips and control solution for the latest information on product specifications and limitations.

Specifications	
Blood volume	
Sample type	
Measuring time	
Measurement range	Refer to the test strip package insert.
Test strip storage conditions	
System operating conditions	
Relative humidity operating range	
Calibration	Plasma equivalent
Meter storage conditions	Temperature: -13–158 °F
Memory capacity	500 blood glucose results and 20 control results with time and date
Automatic power off	2 minutes
Power supply	One 3-volt lithium battery (coin cell type CR2032)
Display	LCD
Dimensions	94 × 52 × 21 mm (LWH)
Weight	Approx. 59 g (with battery)
Construction	Hand-held
Protection class	III
Meter type	The Accu-Chek Aviva meter is suitable for continuous operation.
Control solution storage conditions	Refer to the control solution package insert.

Electromagnetic Compatibility – This meter meets the electromagnetic immunity requirements as per EN ISO 15197 Annex A. The chosen basis for electrostatic discharge immunity testing was basic standard IEC 61000-4-2. In addition, the meter meets the electromagnetic emissions requirements as per EN 61326. The meter's electromagnetic emission is thus low. Interference from the meter to other electrically-driven equipment is not anticipated.

Performance Analysis – Refer to the test strip package insert.

Test Principle – Refer to the test strip package insert.

Product Safety Information

WARNING

- Strong electromagnetic fields may interfere with the proper operation of the meter. Do not use the meter close to sources of strong electromagnetic radiation.
- To avoid electrostatic discharge, do not use the meter in a very dry environment, especially one in which synthetic materials are present.

Travel Documentation

If you are traveling on a commercial airline, you may be required to provide documentation certifying that this meter meets environmental conditions and test procedures for Airborne Equipment (RTCA/DO-160F) section 21 Emission of Radio Frequency Energy. Visit accul-cheek.com or contact the Accu-Chek Customer Care Service Center at 1-800-858-8072 to obtain a copy of the document.

Discarding the Meter, Test Strips, Lancing Devices, Lancets, and Batteries

WARNING

- Any product coming in contact with blood is considered contaminated (potentially infectious).*
- During normal testing, any blood glucose meter may come in contact with blood.
- Lancing devices may also be considered sharps. Disposal of sharps is regulated by law in many jurisdictions.

Roche is committed to recycling and sustainability. Comply with any laws or ordinances relating to the disposal of sharps and/or contaminated products. Contact your local health department or other appropriate authorities for proper handling and disposal of used meters, used test strips, used lancets, and used batteries. Consider the following points when discarding used testing materials:

Consider recycling the meters and batteries at an appropriate facility. Be aware the meter is potentially hazardous electronics scrap (e-scrap) and should be disposed of accordingly. The batteries are potentially hazardous also and should be discarded accordingly.

Disinfect the meter before recycling or discarding.

*29 CFR 1910.1030 – Bloodborne pathogens

Explanation of Symbols

	Caution, refer to safety-related notes in the instructions for use accompanying this product.
	3-volt coin cell type CR2032
	Global Trade Item Number
	Serial number
	Keep new and used batteries away from children.

Warranty

Accu-Chek Aviva Meter 30-day Money-back Guarantee for Qualifying Consumers

Roche offers qualifying consumers that purchase an Accu-Chek Aviva meter, a 30-day money back guarantee. If you are not fully satisfied with your Accu-Chek Aviva meter, contact the Accu-Chek Customer Care Service Center toll-free at 1-800-858-8072 to determine whether you qualify to receive a full refund within 30 days of purchase. If you are covered under Medicare, Medicaid, other federal/state programs, or private insurance you are NOT eligible for this money-back offer. Consumers affected by this exclusion may instead request a different Accu-Chek meter/system. The refund will be limited to the amount paid by you net of any rebates. You must have a copy of the dated itemized purchase receipt and the original packaging to obtain this refund.

Accu-Chek Aviva Meter Limited 3-Year Warranty

Roche warrants to the original purchaser of the meter that your Accu-Chek Aviva meter will be free from defects in materials and workmanship for three years from the date of purchase. If, during this 3-year period, the meter does not work properly because of a defect in materials or workmanship, Roche will replace it with a new Accu-Chek Aviva meter or equivalent product free of charge. The warranty on the replacement meter will expire on the date of the original warranty expiration or 90 days after the shipment of a replacement system, whichever period is longer. The purchaser's exclusive remedy with respect to the Accu-Chek Aviva meter shall be replacement.

This warranty does not apply to the performance of an Accu-Chek Aviva meter that has been damaged by accident or has been altered, misused, tampered with, or abused in any way. Roche will handle meters that show damage or abuse according to its Non-Warranty Service Policy described below.

THE ABOVE WARRANTY IS EXCLUSIVE OF ALL OTHER WARRANTIES, AND ROCHE MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING WITHOUT LIMITATION, THE IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE BE LIABLE TO THE PURCHASER OR ANY OTHER PERSON FOR ANY INCIDENTAL, CONSEQUENTIAL, INDIRECT, SPECIAL, OR PUNITIVE DAMAGES ARISING FROM OR IN ANY WAY CONNECTED WITH THE PURCHASE OR OPERATION OF THE METER OR ITS PARTS. NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, IF ANY IS IMPLIED FROM THE SALE OF THE METER, SHALL EXTEND FOR A LONGER DURATION THAN THREE YEARS FROM THE DATE OF PURCHASE.

Some states do not allow limitations on how long an implied warranty will last or the exclusion of incidental or consequential damages, so the above limitation and exclusion may not apply to you. This warranty gives you specific legal rights, which vary from state to state.

Non-Warranty Service Policy

Roche Non-Warranty Service Policy applies to meters where the above warranty has not become effective, has become inapplicable, or has expired. Roche will replace, at its option, meters returned to it for a service charge (not to exceed \$35). Replacement will be with the same or similar product. Replacement meters will be warranted for a period of 90 days from shipment under a limited warranty providing for replacement of parts and labor at no charge.

Warranty and Service Instructions

All requests for return of Accu-Chek Aviva meters under the above warranty or service policy must be made to the Accu-Chek Customer Care Service Center. You will be mailed a return authorization label, which must be affixed to your carton for shipping the system to Roche. Cartons received without this label will be returned to you at your expense.

Customers experiencing difficulties should review the troubleshooting information in Chapter 7 of this booklet. Further inquiries should be directed to the Accu-Chek Customer Care Service Center.

Be sure to register your warranty online at accu-chek.com/register for the Accu-Chek Aviva Plus system.

8 Technical Information

Limited License

WARNING

CAUTION – A RESTRICTED LICENSE LIMITS USE OF THE ACCU-CHEK AVIVA PLUS SYSTEM IN THE UNITED STATES – READ CAREFULLY THE LIMITATIONS RECITED BELOW.

The Accu-Chek Aviva Plus system (meter, including the test strips) and its use are protected by U.S. Patent Nos. 7,276,146 (expires 4-October-2022); 7,276,147 (expires 4-October-2022); 8,298,401 (expires 4-October-2022); 8,303,801 (expires 4-October-2022); 8,329,026 (expires 4-October-2022); 7,452,457 (expires 2-May-2026); 7,488,601 (expires 1-February-2026); 7,569,126 (expires 28-December-2026); and 7,604,721 (expires 12-August-2026). A license to use the Accu-Chek Aviva Plus system is required until the expiration of the last-to-expire patent listed above and is only granted when the Accu-Chek Aviva meter is used with the Accu-Chek Aviva Plus test strips.

Accu-Chek Aviva Plus test strips are specifically manufactured for operation with the Accu-Chek Aviva meter. Use of other test strips supplied by another manufacturer may prevent or impair the proper function of the Accu-Chek Aviva Plus system.

Using the Accu-Chek Aviva Plus system indicates your acceptance of the restricted license to use the Accu-Chek Aviva Plus system only with Accu-Chek Aviva Plus test strips. Further, if you have purchased an Accu-Chek Aviva monitoring kit or an Accu-Chek Aviva meter that includes this restricted license, then this restricted license applies regardless of any additional offers found in Accu-Chek Aviva Plus test strip packages. If you do not agree to the terms and conditions of the restricted license, you may return, at the place of purchase, the unused Accu-Chek Aviva Plus system for a full refund. If you have any questions, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

Except where prohibited by statute, all warranties covering the Accu-Chek Aviva Plus system are voided by use of the Accu-Chek Aviva Plus system with any test strips other than Accu-Chek Aviva Plus test strips.

Patent Information

U.S. Pat.: <http://www.roche-diagnostics.us/patents>

Additional Supplies

Test Strips: Accu-Chek Aviva Plus test strips

Control Solution: Accu-Chek Aviva 2 level control solutions

Lancets: Accu-Chek Softclix 100-ct. lancets

AST Cap: Available upon request from Accu-Chek Customer Care Service Center at 1-800-858-8072.

References

- ¹ FDA Public Health Notification: "Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens: Initial Communication, (2010). Update 11/29/2010." <http://wayback.archive-it.org/7993/20161022010458/http://www.fda.gov/MedicalDevices/Safety/AlertsandNotices/ucm224025.htm>. Accessed September 19, 2018.
- ² CDC Clinical Reminder: "Use of Fingerstick Devices on More than One Person Poses Risk for Transmitting Bloodborne Pathogens, (2010)." <http://www.cdc.gov/injectionsafety/Fingerstick-DevicesBGM.html>. Accessed September 19, 2018.
- ³ Healthcare Infection Control Practices Advisory Committee (HICPAC), William A. Rutala, Ph.D., M.P.H., and David J. Weber, M.D., M.P.H. Centers for Disease Control and Prevention, 2008. "Guideline for Disinfection and Sterilization in Healthcare Facilities." Atlanta. <https://www.cdc.gov/infectioncontrol/pdf/guidelines/disinfection-guidelines.pdf>. Accessed September 19, 2018.

A

alternate site testing 23

B

battery, changing 43

battery, installing 43

battery type 43, 47

beeper, setting 31

blood glucose results, unusual 25

blood glucose test 21

button, power/set 7, 11

C

comfort dial 17, 19

computer, transferring results to 36

control results, understanding

out-of-range 16

control solution 8, 13

control test, performing 14

D

display check 11

display messages 44

E

error messages 45

G

general marker 24

H

high blood glucose 26

hyperglycemia 26

hypoglycemia 26

hypoglycemic alert, setting 35

L

lancet, inserting 18

lancing device, cleaning and disinfecting 40

lancing device, using 19

low blood glucose 26

M

maintenance, meter 43

memory, meter 27

meter, cleaning and disinfecting 37

P

post-meal marker 24

post-meal test reminder 24

post-meal test reminder, setting 32

pre-meal marker 24

product limitations 47

product safety information 48

product specifications 47

S

safety information 3

settings, meter 27

supplies 54

symbols 49

symptoms, hypoglycemia/hyperglycemia 26

Index

T

technical information 47
test reminders, setting 33
test strips 8, 10
time and date, setting 30
troubleshooting 44

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ACCU-CHEK, ACCU-CHEK AVIVA, and SOFTCLIX are trademarks of Roche.
All other product names and trademarks are the property of their respective owners.
Roche Diabetes Care, Inc.