ENGLISH

Start Here


Quick Start Guide

For in vitro diagnostic use only

For use with the MiniMed™ 770G and MiniMed™ 780G systems with **Bluetooth®** wireless technology

WARNING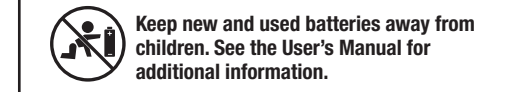

- **This Quick Start Guide does not replace the User's Manual for your Accu-Chek Guide Link blood glucose meter. The User's Manual contains important cleaning and disinfecting procedures and additional information.**
- **The meter and lancing device are for single patient use. Do not share them with anyone, including other family members! Do not use on multiple patients!**
- **All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.**

NOTE


- **Always have a spare set of batteries.**
- **Battery life may vary due to factors such as temperature and battery manufacturer.**

Meter Overview


- 1 Display
 - 2 Back Button
 - 3 Up Arrow and Down Arrow Buttons
 - 4 Power/Set/OK Button
 - 5 Test Strip Slot with Light
 - 6 Battery Door
 - 7 Micro USB Port*
 - 8 Test Strip Ejector
- *USB functionality is not available for this meter.

Set the Meter Language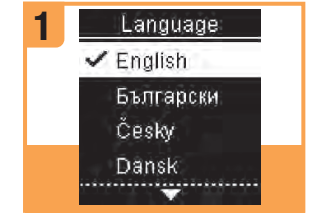

Turn the meter on by briefly pressing **OK**. **Language** appears. Press **Left** or **Right** to highlight the desired language.

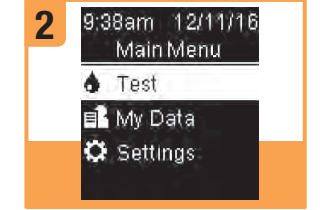
Press **OK** to set the language and return to **Main Menu**.

Pair the Meter and Insulin Pump


The Accu-Chek Guide Link meter can automatically send blood glucose results to the MiniMed™ 770G and MiniMed™ 780G systems with Bluetooth® wireless technology. This feature eliminates the need to manually enter your blood glucose result on the insulin pump. Refer to the MiniMed™ 770G and MiniMed™ 780G systems User Guide for pairing instructions.


Set up the Lancing Device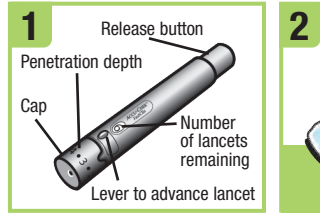

Familiarize yourself with the Accu-Chek FastClix lancing device and the lancet drum.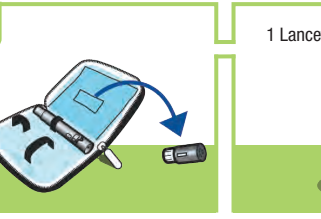


Get the lancet drum from the pocket in the carry case.

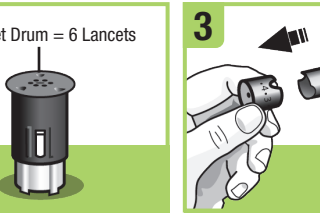
NOTE
Each lancet drum contains 6 sterile lancets. You can perform 6 tests before replacing it.

NOTE
Do not remove the lancet drum until all 6 lancets are used since the drum cannot be reused once it has been removed from the lancing device.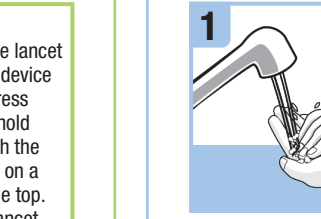


Place the cap back on the lancing device. Make sure the notch on the cap lines up with the notch on the lancing device.

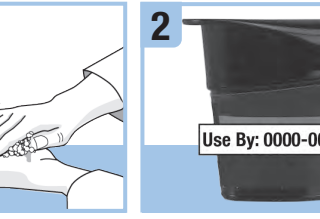Adjust the lancet depth to a comfortable penetration level. Start at 2. For tougher skin, dial to a higher number.


Using the lancing device: By pressing the release button all the way down, the lancing device primes and pricks with 1 click.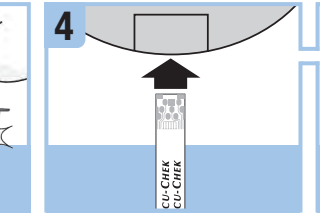

When ready to test...

Perform a Blood Glucose Test

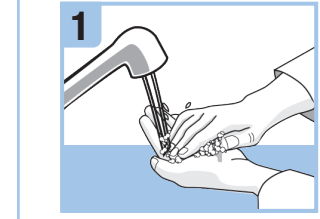
Wash and dry your hands thoroughly **before** and **after** you perform a blood glucose test.

Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.


Remove a test strip from the test strip container. Close the cap tightly.

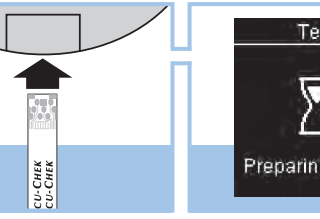
Insert the metallic end of the test strip into the meter.


The meter turns on. **Preparing to test** appears.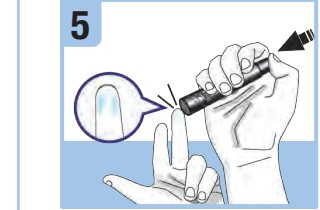


Press the lancing device firmly against the side of your fingertip and press the release button all the way down to prick your finger.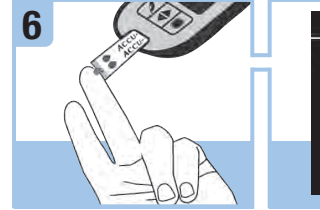

Touch the **yellow edge** of the test strip to the blood drop. Do not put blood on top of the test strip. Remove your finger from the test strip when **Analyzing** appears. Failure to move your finger away from the test strip could give inaccurate test results.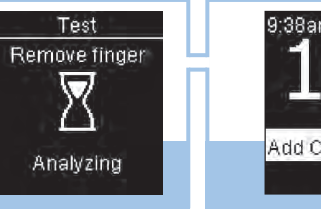

The test result appears on the display in less than 4 seconds. Press **OK** to automatically send result to a paired pump. If **OK** is not pressed, there will be a delay seeing the blood glucose result on the paired pump.

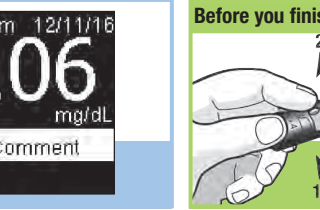Prepare the lancing device for the next test. Slide the lever over and back once to advance to the next lancet. The window shows the number of lancets remaining.

Roche Diabetes Care, Inc.
9115 Hague Road
Indianapolis, IN 46256
www.accu-chek.com

ACCUCHEK, ACCUCHEK GUIDE, and FASTCLIX are trademarks of Roche.
The **Bluetooth®** word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Roche is under license.
MiniMed is a registered trademark of Medtronic, Inc.

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© 2021 Roche Diabetes Care