Start Here

Quick Start Guide


For in vitro diagnostic use only

For use with the MiniMed™ 770G and MiniMed™ 780G systems with Bluetooth® wireless technology

Medtronic

Roche

WARNING

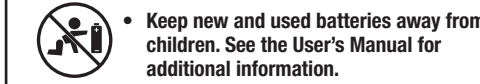- Keep new and used batteries away from children. See the User's Manual for additional information.
- This Quick Start Guide does not replace the User's Manual for your Accu-Chek Guide Link blood glucose meter. The User's Manual contains important cleaning and disinfecting procedures and additional information.
- The meter and lancing device are for single patient use. Do not share them with anyone, including other family members! Do not use on multiple patients!
- All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.

NOTE

- Always have a spare set of batteries.
- Battery life may vary due to factors such as temperature and battery manufacturer.

Meter Overview


- 1 Display
- 2 Back Button
- 3 Up Arrow and Down Arrow Buttons
- 4 Power/Set/OK Button
- 5 Test Strip Slot with Light
- 6 Battery Door
- 7 Micro USB Port*
- 8 Test Strip Ejector

*USB functionality is not available for this meter.

Set the Meter Language

Turn the meter on by briefly pressing **OK**. **Language** appears. Press **Up Arrow** or **Down Arrow** to highlight the desired language.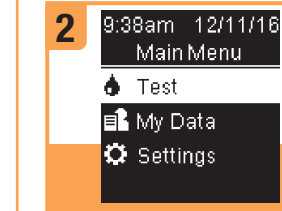


Press **OK** to set the language and return to **Main Menu**.

Pair the Meter and Insulin Pump


The Accu-Chek Guide Link meter can automatically send blood glucose results to the MiniMed™ 770G and MiniMed™ 780G systems with Bluetooth® wireless technology. This feature eliminates the need to manually enter your blood glucose result on the insulin pump. Refer to the MiniMed™ 770G and MiniMed™ 780G systems User Guide for pairing instructions.

Set up the Lancing Device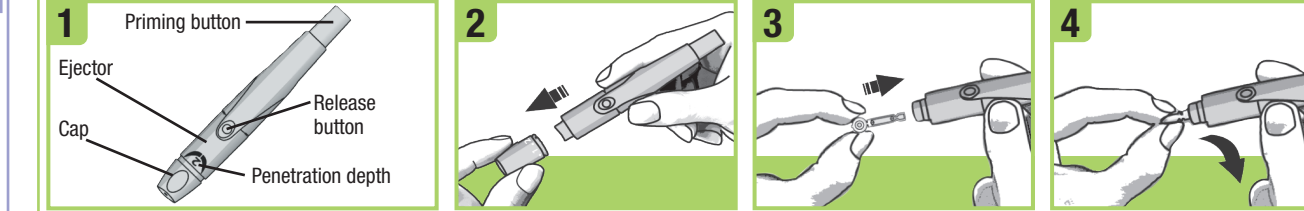

Familiarize yourself with the Accu-Chek Softclix lancing device.

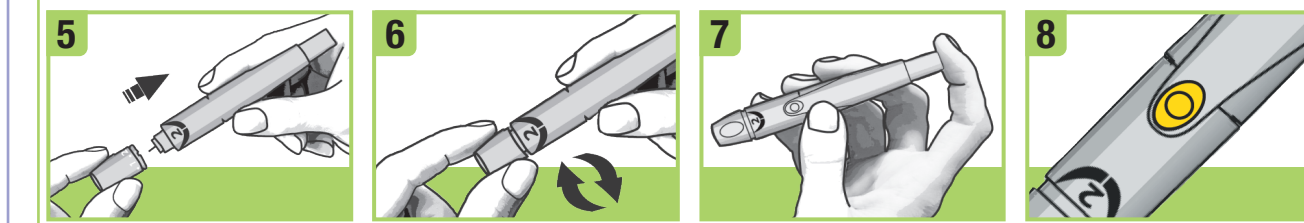Place the cap back on the lancing device. Make sure the notch on the cap lines up with the notch on the lancing device.

Adjust the lancet depth to a comfortable penetration depth. Start at 2. For tougher skin, rotate to a higher number.

Press the priming button in as far as it will go, like a pen. Do not press the release button while pressing the priming button.

The release button turns yellow when the lancing device is ready. Set the lancing device aside until you are ready to perform a fingerstick.

Perform a Blood Glucose Test


Wash and dry your hands thoroughly **before** and **after** you perform a blood glucose test.

Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.

Remove a test strip from the test strip container. Close the cap tightly.

Insert the metallic end of the test strip into the meter.

The meter turns on. **Preparing to test** appears.

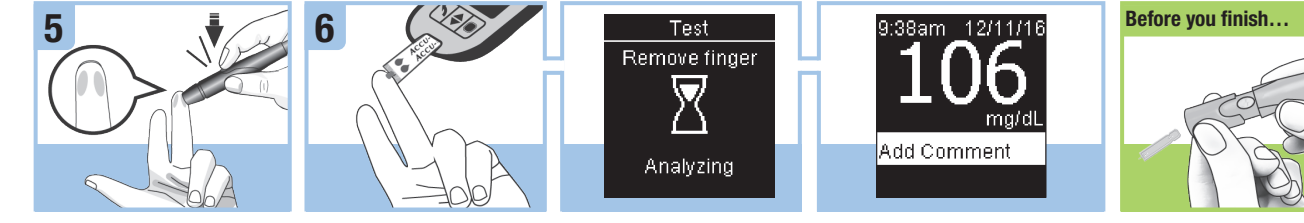Press the lancing device firmly against the side of your fingertip and press the yellow release button to prick your finger.

Touch the **yellow edge** of the test strip to the blood drop. Do not put blood on top of the test strip. Remove your finger from the test strip when **Analyzing** appears.

The test result appears on the display in less than 4 seconds. Press **OK** to automatically send the test result to a paired pump. If **OK** is not pressed, there will be a delay seeing the blood glucose result on the paired pump.

Before you finish... Remove the lancing device cap. Point the lancet away from you. Slide out the ejector to release the lancet.

Roche Diabetes Care, Inc.
9115 Hague Road
Indianapolis, IN 46256, USA
www.accu-chek.com

ACCUCHEK, ACCUCHEK GUIDE, and SOFTCLIX are trademarks of Roche.
The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Roche is under license.
MiniMed is a registered trademark of Medtronic, Inc.

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© 2022 Roche Diabetes Care
MiniMed es una marca registrada de Medtronic, Inc.