# LET'S GET STARTED

Create your Accu-Chek account to access exclusive offers and resources

Create your Accu-Chek account

accu-chek.com/register

For in vitro diagnostic use only.

## WARNING

- Keep new and used batteries away from children. See the User's Manual for additional information.

- This Quick Start Guide does not replace the User's Manual for your Accu-Chek Guide Me blood glucose meter. The User's Manual contains important cleaning and disinfecting procedures and additional information.
- The meter and lancing device are for single patient use. Do not share them with anyone, including other family members! Do not use on multiple patients!
- All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.
- If you drop the meter or drop the meter with a test strip inserted, the meter and/or test strip could be damaged. Discard the test strip and perform a control test with control solution and a new, unused test strip to ensure the meter and test strips are both working properly. Then repeat the blood glucose test with a new test strip.
- When performing a blood glucose test, remove your finger from the test strip after the test strip is dosed and the flashing hourglass symbol appears on the meter. Failure to move your finger away from the test strip during measurement could give inaccurate test results.

## NOTE

- Always have a spare set of batteries.
- Battery life may vary due to factors such as temperature and battery manufacturer.

## CREATE YOUR ACCU-CHEK ACCOUNT

- Scan the QR code or visit [accu-chek.com/register](http://accu-chek.com/register) to access exclusive offers and resources.


## OVERVIEW


- 1 Power/Set Button
- 2 Display
- 3 Left Arrow Button
- 4 Right Arrow Button
- 5 Test Strip Slot
- 6 Battery Door
- 7 Micro USB Port

## SET THE TIME AND DATE

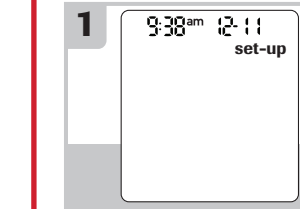
The meter prompts you to set the time and date the first time you turn it on. **Set-up** appears on the display with the hour flashing. Press or to decrease or increase the hour.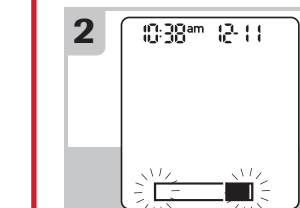


Press to set the hour and move to the next field. Repeat steps 1 and 2 to set minutes, am or pm, month, day, and year. Press until the flashing test strip symbol appears.


## SET UP THE LANCING DEVICE

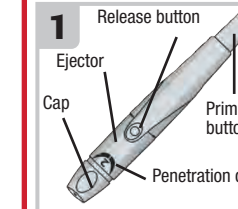
Familiarize yourself with the Accu-Chek Softclix lancing device.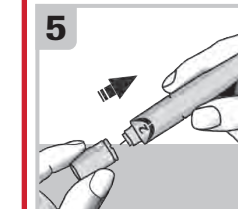


Place the cap back on the lancing device. Make sure the notch on the cap lines up with the notch on the lancing device.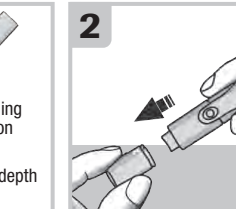


Remove the lancing device cap by pulling it straight off. Do not twist the cap.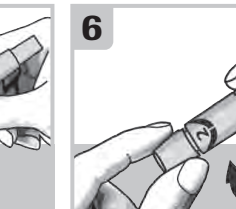


Adjust the lancet depth to a comfortable penetration depth. Start at 2. For tougher skin, rotate to a higher number.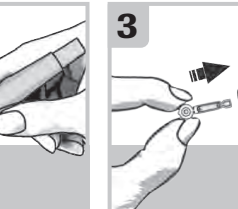


Insert the lancet into the lancing device until it clicks.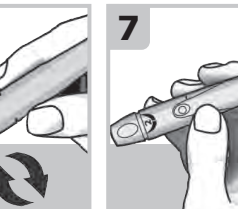

Press the priming button in as far as it will go, like a pen. Do not press the release button while pressing the priming button.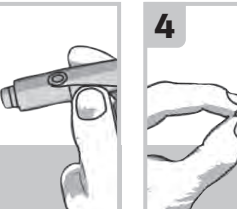


Twist off the lancet's protective cap.

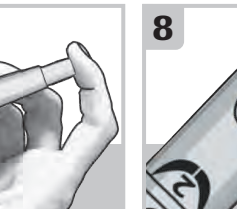
The release button turns yellow when the lancing device is ready. Set the lancing device aside until you are ready to perform a fingerstick.


## PERFORM A BLOOD GLUCOSE TEST

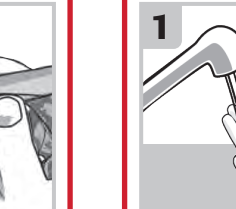
Wash and dry your hands thoroughly **before** and **after** you perform a blood glucose test.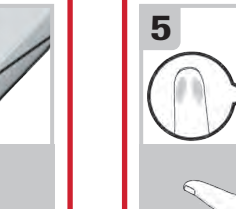

Press the lancing device firmly against the side of your fingertip and press the yellow release button to prick your finger.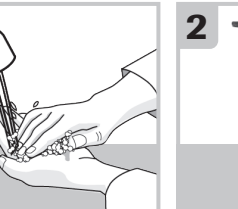


Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.

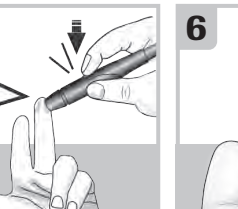Touch the **yellow edge** of the test strip to the blood drop. Do not put blood on top of the test strip.


Remove a test strip from the test strip container. Close the cap tightly.

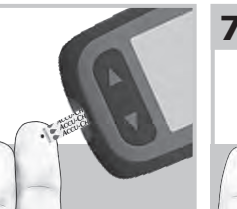
Remove your finger from the test strip when the flashing appears.


Insert the metallic end of the test strip into the meter.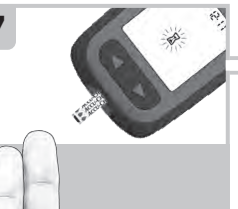

The test result appears on the display in less than 4 seconds.

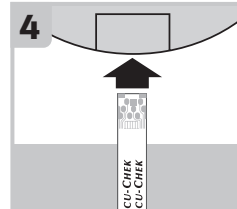
The meter turns on. A flashing drop symbol appears.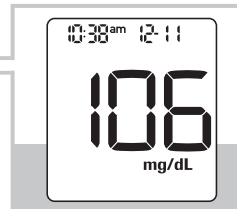


Remove the lancing device cap. Point the lancet away from you. Slide out the ejector to release the lancet.


## CONNECT THE ACCU-CHEK GUIDE ME METER TO THE MYSUGR APP

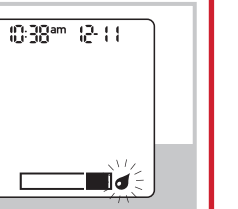Using your smartphone, **download the mySugr app** from the App Store or Google Play and **create your mySugr account**.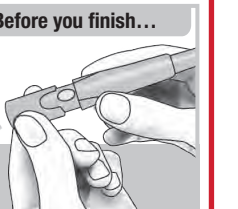

Open the mySugr app, and select **Connections** on the menu. Select **Accu-Chek Guide Me**, and then **Connect now**. Follow the step-by-step directions for pairing.

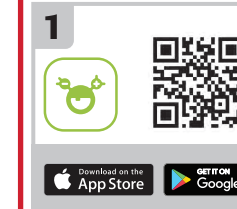Turn on **Bluetooth** wireless technology on your smartphone (if it's not on already).


After your meter is successfully connected to the mySugr app, **data from your device is imported** automatically. Now you can easily keep your diabetes data under control!

## CONNECT THE ACCU-CHEK GUIDE ME METER TO THE MYSUGR APP

Using your smartphone, **download the mySugr app** from the App Store or Google Play and **create your mySugr account**.

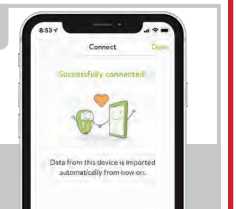After your meter is successfully connected to the mySugr app, **data from your device is imported** automatically. Now you can easily keep your diabetes data under control!

Create your Accu-Chek account

accu-chek.com/register

Roche Diabetes Care, Inc.  
9115 Hague Road  
Indianapolis, IN 46256, USA  
[www.accu-chek.com](http://www.accu-chek.com)

ACCUCHEK, ACCUCHEK GUIDE, ACCUCHEK GUIDE ME, SOFTCLIX, MYSUGR, and the MYSUGR logo are trademarks of Roche.  
APPLE, APPLE logo, and APP STORE are trademarks of Apple Inc. GOOGLE PLAY is a trademark of Google Inc. The *Bluetooth* word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Roche is under license.

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© 2023 Roche Diabetes Care