# ACCU-CHEK® Guide

Blood Glucose Monitoring System

**ENGLISH**

## Start Here

### Quick Start Guide

For in vitro diagnostic use only

**WARNING**

- Keep new and used batteries away from children. See the User's Manual for additional information.
- This Quick Start Guide does not replace the User's Manual for your Accu-Chek Guide blood glucose meter. The User's Manual contains important cleaning and disinfecting procedures and additional information.
- The meter and lancing device are for single patient use. Do not share them with anyone, including other family members! Do not use on multiple patients!
- All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.

**NOTE**

- Always have a spare set of batteries.
- Battery life may vary due to factors such as temperature and battery manufacturer.

**ACCU-CHEK®**

### Overview

1 Display  
2 Back Button  
3 Up Arrow and Down Arrow Buttons  
4 Power/Set/OK Button  
5 Test Strip Slot with Light  
6 Battery Door  
7 Micro USB Port  
8 Test Strip Ejector

### Set the Meter Language

1 Language  
✓ English  
简体中文  
繁體中文  
Dansk

Turn the meter on by briefly pressing OK. Language appears. Press ▲ or ▼ to highlight the desired language.

2 9:38am 12/11/15 Main Menu  
Test  
My Data  
Settings

Press OK to set the language and return to Main Menu.

### Set up the Lancing Device

- Release button  
Cap  
Penetration depth
- Get a lancet from the pocket in the carrying case.
- Remove the cap by pulling it straight off. Do not twist the cap.
- Insert the lancet into the lancing device until it clicks.
- Twist off the lancet's protective cap.

Familiarize yourself with the Accu-Chek Softclix lancing device. Check the Use By date on the test strip container. Do not use test strips past the Use By date. Insert the metallic end of the test strip into the meter. The meter turns on. Preparing to test appears.

### Perform a Blood Glucose Test

- Ejector  
Cap  
Priming button
- Use By: 0000-00-00
- Check the Use By date on the test strip container. Do not use test strips past the Use By date.
- Remove a test strip from the test strip container. Close the cap tightly.
- Turn on Bluetooth® on your smartphone (if it's not on already).
- Wash and dry your hands before performing a blood glucose test.
- Turn on Bluetooth® on your smartphone (if it's not on already).
- Replace the cap on the lancing device. Make sure the notch on the cap lines up with the notch on the lancing device.
- Adjust the lancet depth to a comfortable penetration level. Start at 2. For tougher skin, dial to a higher number.
- Press the priming button firmly against the side of your fingertip and return to Main Menu.
- Touch the yellow edge of the test strip to the blood drop. Remove your finger from the test strip when Analyzing appears. Do not put the test strip in less than 4 seconds.
- The test result appears on the display in less than 4 seconds.
- Remove the lancing device cap. Point the lancet away from you. Slide out the ejector to release the lancet.
- Open the mySugr app, and select Connections on the menu. Select Accu-Chek Guide, and then Connect now.
- After your meter is successfully connected to the mySugr app, data from your device is imported automatically. Now you can easily keep your diabetes data under control!

To find diabetes management apps that are compatible with your meter, visit [www.accu-check.com](http://www.accu-check.com).

Roche Diabetes Care, Inc.  
9115 Hague Road  
Indianapolis, IN 46256  
[www.accu-check.com](http://www.accu-check.com)

ACCU-CHEK, ACCU-CHEK GUIDE, SOFTCLIX, and MYSUGR are trademarks of Roche.

APPLE, APPLE logo, and APP STORE are trademarks of Apple Inc.

GOOGLE PLAY is a trademark of Google Inc.

The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Roche is under license.

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© 2021 Roche Diabetes Care
APPLE, el logotipo de APPLE y APP STORE son marcas comerciales de Apple Inc.
GOOGLE PLAY es una marca comercial de Google Inc.