
**WARNING**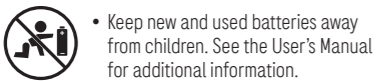

Keep new and used batteries away from children. See the User's Manual for additional information.

- This Quick Start Guide does not replace the User's Manual for your Accu-Chek Guide blood glucose meter. The User's Manual contains important cleaning and disinfecting procedures and additional information.
- The meter and lancing device are for single patient use. Do not share them with anyone, including other family members! Do not use on multiple patients!
- All parts of the kit are considered biohazardous and can potentially transmit infectious diseases from bloodborne pathogens, even after you have performed cleaning and disinfecting.

**NOTE**

- Always have a spare set of batteries.
- Battery life may vary due to factors such as temperature and battery manufacturer.

**CREATE YOUR ACCU-CHEK ACCOUNT**

1 Scan the QR code or visit [accu-chek.com/register](http://accu-chek.com/register) to access exclusive offers and resources.

**OVERVIEW**


- |                                   |                              |
|-----------------------------------|------------------------------|
| 1 Display                         | 5 Test Strip Slot with Light |
| 2 Back Button                     | 6 Battery Door               |
| 3 Up Arrow and Down Arrow Buttons | 7 Micro USB Port             |
| 4 Power/Set/OK Button             | 8 Test Strip Ejector         |

**SET THE METER LANGUAGE**


Turn the meter on by briefly pressing **OK**. **Language** appears. Press **Left** or **Right** to highlight the desired language.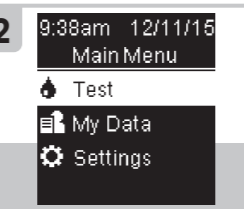


Press **OK** to set the language and return to **Main Menu**.


**SET UP THE LANCING DEVICE**

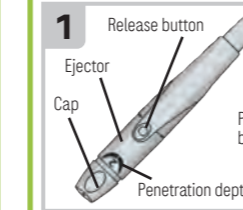
Familiarize yourself with the Accu-Chek Softclix lancing device.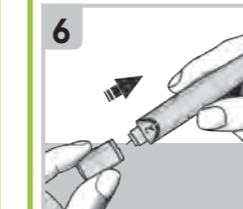


Replace the cap on the lancing device. Make sure the notch on the cap lines up with the notch on the lancing device.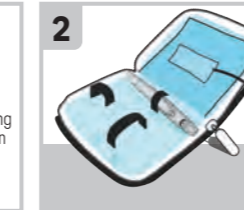


Get a lancet from the pocket in the carrying case.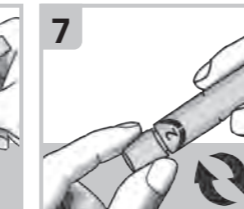


Adjust the lancet depth to a comfortable penetration depth. Start at 2. For tougher skin, rotate to a higher number.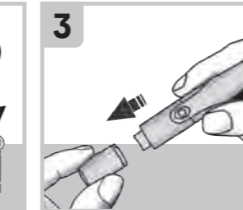


Remove the lancing device cap by pulling it straight off. Do not twist the cap.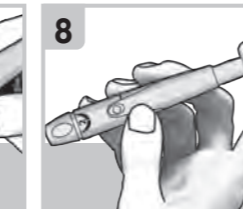


Press the priming button in as far as it will go, like a pen. Do not press the release button while pressing the priming button.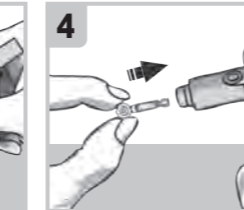


Insert the lancet into the lancing device until it clicks.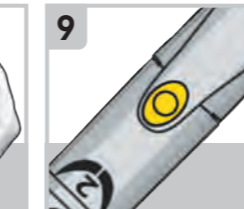

The release button turns yellow when the lancing device is ready. Set the lancing device aside until you are ready to perform a fingerstick.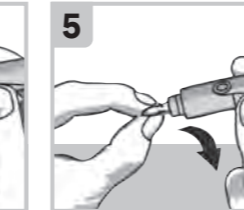


Twist off the lancet's protective cap.

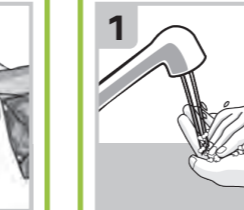
Press the lancing device firmly against the side of your fingertip and press the yellow release button to prick your finger.


**PERFORM A BLOOD GLUCOSE TEST**

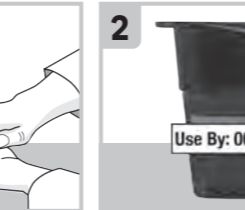
Wash and dry your hands thoroughly **before** and **after** you perform a blood glucose test.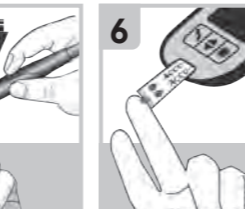


Press the lancing device firmly against the side of your fingertip and press the yellow release button to prick your finger.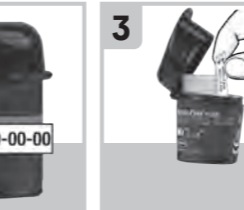


Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.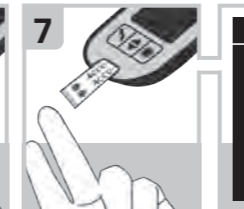

Touch the **yellow edge** of the test strip to the blood drop. Do not put blood on top of the test strip.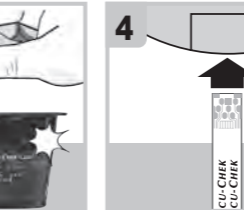


Remove a test strip from the test strip container. Close the cap tightly.

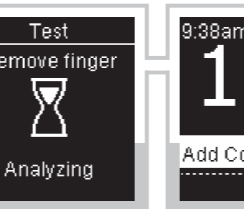Remove your finger from the test strip when **Analyzing** appears.

The meter turns on. **Preparing to test** appears.

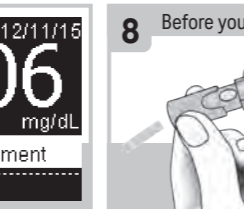The test result appears on the display in less than 4 seconds.

The meter turns on. **Preparing to test** appears.

Remove the lancing device cap. Point the lancet away from you. Slide out the ejector to release the lancet.

**CONNECT THE ACCU-CHEK GUIDE METER TO THE MYSUGR APP**


Using your smartphone, **download the mySugr app** from the App Store or Google Play and **create your mySugr account**.

Open the mySugr app, and select **Connections** on the menu. Select **Accu-Chek Guide**, and then **Connect now**. Follow the step-by-step directions for pairing.

Turn on **Bluetooth** wireless technology on your smartphone (if it's not on already).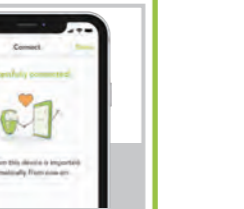

After your meter is successfully connected to the mySugr app, **data from your device is imported** automatically. Now you can easily keep your diabetes data under control!

Create your Accu-Chek account

accu-chek.com/register

Roche Diabetes Care, Inc.  
9115 Hague Road  
Indianapolis, IN 46256, USA  
[www.accu-chek.com](http://www.accu-chek.com)

ACCUCHEK, ACCU-CHEK GUIDE, SOFTCLIX, MYSUGR, and the MYSUGR logo are trademarks of Roche.

APPLE, APPLE logo, and APP STORE are trademarks of Apple Inc.

GOOGLE PLAY is a trademark of Google Inc.

The **Bluetooth** word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Roche is under license.

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© 2024 Roche Diabetes Care