

ACCU-CHEK® Nano

Roche

English

Start Here

Quick Start Guide

Watch at accu-cheek.com

WARNING
This Quick Start Guide does not replace the Owner's Booklet for your Accu-Chek Nano blood glucose meter. The Owner's Booklet contains important handling instructions and additional information.

WARNING
The lancing device is intended for personal use only.

Overview

A. Test Strip Container*
B. Accu-Chek Nano Meter
C. Accu-Chek FastClix Lancing Device*
D. Lancet Drum – Contains 6 lancets*
E. Owner's Booklet

*Some items may not be included in the kit. They are a separate purchase.

Setting up the Lancing Device

- Release button
Set depth
Cap
Number of lancets remaining
Advance to next lancet
- Get the lancet drum from the pocket in the carry case.
- 1 Lancet Drum=6 Lancets
- NOTE**
Each lancet drum contains 6 sterile lancets. You can perform 6 tests before replacing it.
- Remove the cap by pulling it straight off. Do not twist the cap.
- Insert a test strip from the test strip container. Close the cap tightly.
- WARNING**
You must not insert the lancet drum into the lancing device and simultaneously press the release button or hold the lancing device with the release button resting on a surface such as a table top. This could release a lancet and inadvertently cause injury.

- Wash and dry your hands before and after you perform a blood glucose test.
- Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.
- Remove a test strip from the test strip container. Close the cap tightly.
- Insert the test strip into the meter in the direction of the arrows until the meter beeps again and a flashing blood drop symbol appears.
- NOTE**
Do not remove the lancet drum until all 6 lancets are used, since the lancet drum cannot be reused once it has been removed from the lancing device.
- Replace the cap on the lancing device. Make sure that the notch on the cap lines up with the notch on the lancing device.
- NOTE**
Push the lancet drum all the way in until it clicks firmly into place.
- Adjust the lancet depth to a comfortable penetration level. Start at 2. For tougher skin, dial to a higher number.
- Using the lancing device: By pressing the release button, the lancing device primes and pricks with 1 click.
- When ready to test...**
- Before you finish...**

Performing a Blood Glucose Test

- Wash and dry your hands before and after you perform a blood glucose test.
- Check the **Use By** date on the test strip container. Do not use test strips past the **Use By** date.
- Remove a test strip from the test strip container. Close the cap tightly.
- Insert the test strip into the meter in the direction of the arrows until the meter beeps again and a flashing blood drop symbol appears.
- The meter beeps again and a flashing blood drop symbol appears.
- The test result appears on the display after 5 seconds.
- NOTE**
If you see any other error on the display, contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.
- NOTE**
Do not remove the lancet drum until all 6 lancets are used, since the lancet drum cannot be reused once it has been removed from the lancing device.
- Replace the cap on the lancing device. Make sure that the notch on the cap lines up with the notch on the lancing device.
- NOTE**
Push the lancet drum all the way in until it clicks firmly into place.
- Hold the lancing device firmly against the edge of your fingertip and press the release button until it clicks to prick your finger.
- Touch the end of the test strip to the blood drop.
- The meter beeps and flashes when there is enough blood in the test strip.
- Prepare the lancing device for the next test. Slide the lever back and forth once to advance to the next lancet. The window shows the number of lancets remaining.
- Contact the Accu-Chek Customer Care Service Center at 1-800-858-8072.

Error Messages

- E-8 The temperature is above or below the proper range for the system. Refer to the test strip package insert for system operating conditions.
- E-1 The test strip may be damaged or not properly inserted. Remove and reinsert the test strip, or replace if damaged.
- E-3 Your blood glucose error has occurred. If your test result matches how you feel, contact your healthcare professional immediately. If your test result does not match how you feel, repeat the blood glucose test. See the Owner's Booklet, Chapter 4, Unusual Blood Glucose Results. If the E-3 code still appears for your blood glucose test, your blood glucose result may be extremely high and above the system's reading range. Contact your healthcare professional immediately. If the second test result does not match how you feel, perform a control test with the control solution and a new test strip. If the control result is within the acceptable range, review the proper testing procedure and repeat the blood glucose test with a new test strip. If the control result is not within the acceptable range, see the Owner's Booklet, Chapter 2, Understanding Out-of-Range Control Results.
- E-4 Not enough blood or control solution was drawn into the test strip for measurement or was applied after the test had started. Discard the test strip and repeat the test.
- E-6 Blood or control solution was applied to the test strip before the flashing drop symbol appeared on the display. Discard the test strip and repeat the test.
- E-7 An electronic error occurred or, in rare cases, a used test strip was removed and reinserted. Turn the meter off and on, or take the batteries out for 20 seconds and reinsert them. Perform a blood glucose or control test. If the problem persists, contact the Accu-Chek Customer Care Service Center.

ACCU-CHEK, ACCU-CHEK NANO, ACCU-CHEK NANO SMARTVIEW, and FASTCLIX are trademarks of Roche.

© 2016 Roche Diabetes Care
06333656005-0416

Empiece aquí

Guía rápida de instrucciones




Vé en accu-cheek.com

ADVERTENCIA


Esta Guía rápida de instrucciones no reemplaza al Manual del usuario del medidor de glucemia Accu-Chek Nano. El Manual del usuario contiene importantes instrucciones de manejo así como información adicional.

ACCU-CHEK®


Visión general

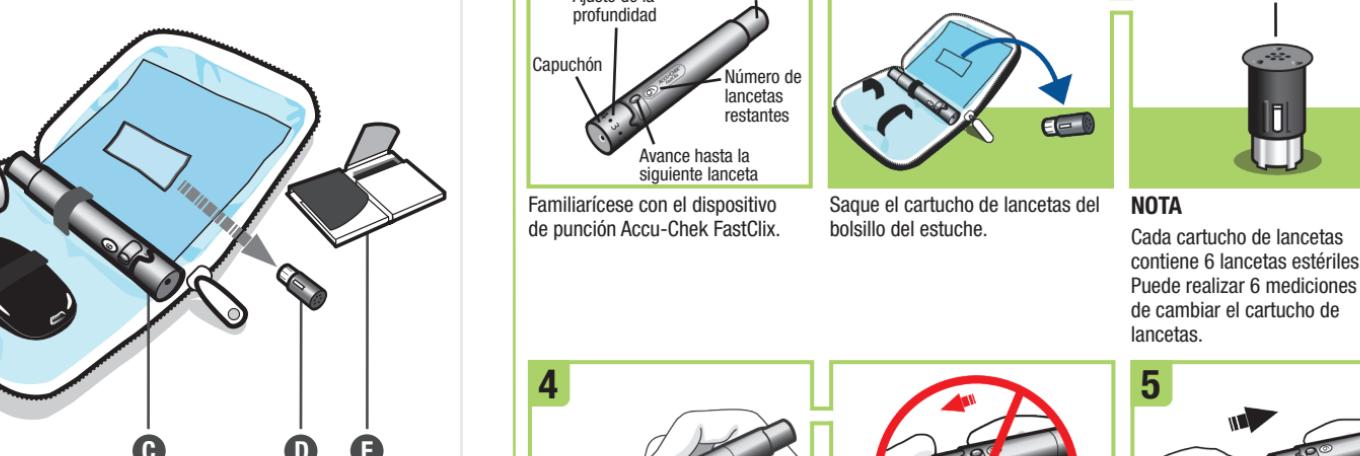


Ajustar el dispositivo de punción

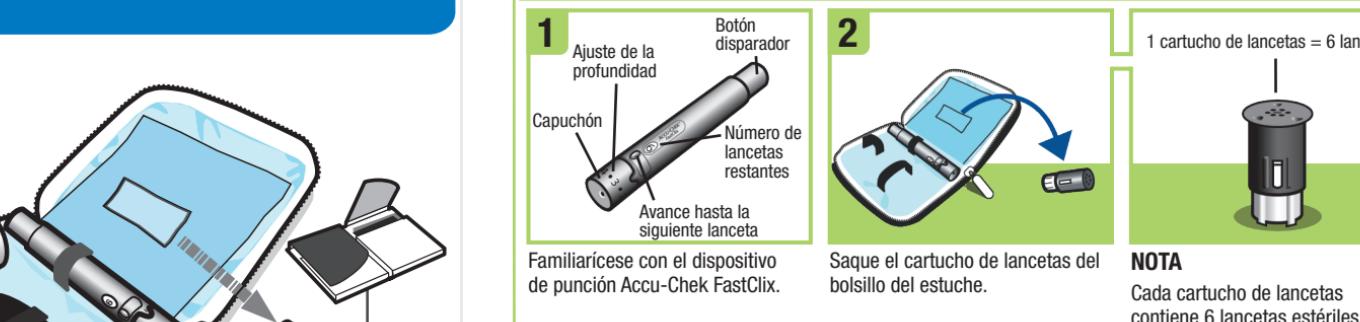

Realizar una medición de glucemia

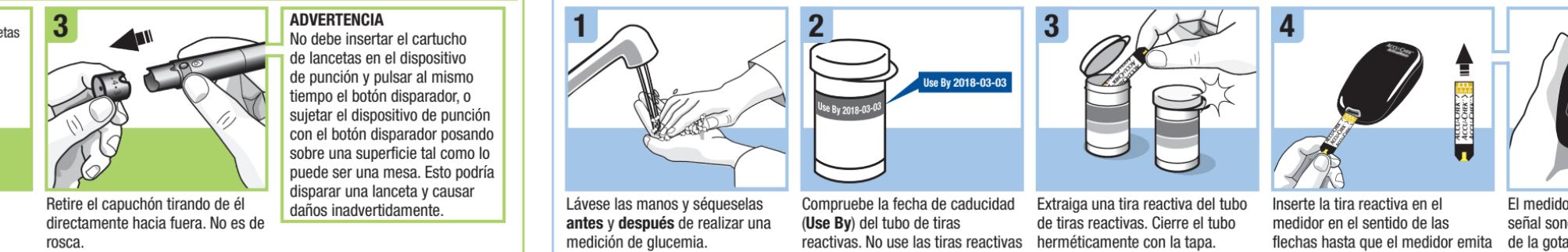

Después de realizar la medición

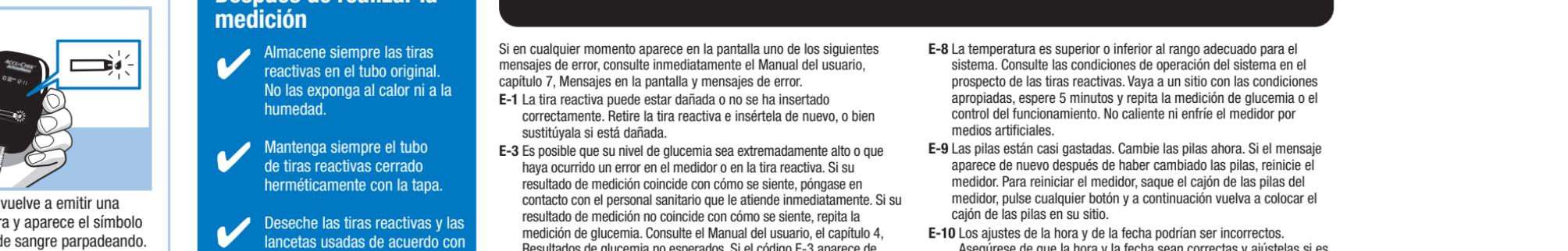

Mensajes de error

