

Diabetes data and detailed reports at your fingertips to share with your healthcare professional

Getting started with the mySugr app

- 1 Download the [mySugr app](#) on the App Store or Google Play.
- 2 Create your mySugr account.
- 3 Unlock mySugr Pro for free to access the advanced PDF report feature and more:



Pair your Accu-Chek® meter for unlimited access to Pro. Go to “Connections” on the menu to start the [pairing process](#).




If you do not have an Accu-Chek® meter, get 6 months of Pro for free by visiting <http://shop.mysugr.com/en/voucher> and entering the voucher code **2020-2020-2020**.
(Code may be redeemed until September 30, 2020.)

- 4 Import your BG values automatically from your Accu-Chek® meter and/or log your data manually.


Stay connected: share mySugr reports with your healthcare professional

Share PDF reports detailing 2 weeks or 3 months of your diabetes data. ([Watch tutorial here.](#))


Producing a report and sharing the data

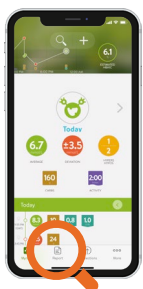


Select “Report” from the menu

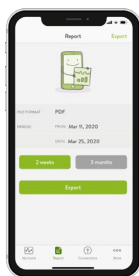

Choose the desired time period and click “Export”

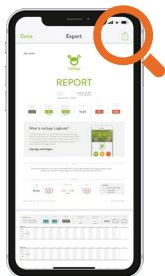

Click the share icon

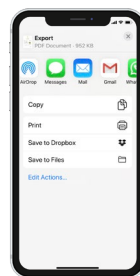

Select the app you wish to use to share the report



For local customer support, contact +1 (855) 337-7847.

Code can only be redeemed one time per user until September 30, 2020. Code will enable mySugr Pro version for 185 days. Void where prohibited by law.

ACCU-CHEK and MYSUGR are trademarks of Roche. ©2020 Roche Diabetes Care, Inc. All rights reserved.

More than 2 million registered users.
Average 4.6 star user experience rating.